Primado2

Attachments

OPERATION
MANUAL

These Attachments are to be used exclusively with our total surgical system Primado2 and Pneumatic Surgical Drilling Machine Primado Air. We recommend that prior to use, you carefully read this Operation Manual regarding "Precaution for Handling and Operation", "Connection method", "Operation method", "Check before Operation" and "Periodical Maintenance Check" so that you can continue using the Attachments in the future.

In addition, keep this Operation Manual in a place where a user can refer to it at any given time.

- In this Operation Manual Bur, Drill, Blade and Rasp are collectively called Cutting Accessory(ies).
- For reprocessing and operation of the Control Unit, Foot Control, Motor or Attachments and applicable Cutting Accessories, refer to each Operation Manual (see Table-1).

Table-1

Operation Manual Title	Operation Manual No.
Primado2 OPERATION MANUAL Control Unit and Foot Controls	OM-SE0021E
Primado2 OPERATION MANUAL Motors	OM-SE0023E
Primado Air OPERATION MANUAL	OM-SM0002E
Primado2 REPROCESSING MANUAL	OM-SZ0911MA
NSK Sterile Cutting Accessories OPERATION MANUAL	OM-Z0147E
NSK Sterile Cutting Accessories OPERATION MANUAL	OM-SZ0031E
NSK Sterile Cutting Accessories For P300 Attachment OPERATION MANUAL	OM-SZ0959MA
CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE	OM-SZ0920MA
CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE	OM-SZ0958MA

Contents

1. User and Intended Purpose / Indication for Each Attachment	2	6. Operation of WPD Series	42
1-1 User and Intended Purpose	2	6-1 Mounting and Removing the Attachment	42
1-2 Indication for Each Attachment	2	6-2 Mounting and Removing the Cutting Accessories (Optional)	42
2. Precautions for Handling and Operation	3	7. Mounting and Removing the Bur Guard	47
3. Product Description	6	8. Check before Operation	47
3-1 Reference Pages and Package Information by Order Code	6	9. Periodical Maintenance Checks	48
3-2 Applicable Product	11	10. Reprocessing	48
3-3 Part Names and Specifications	12	11. Troubleshooting	49
4. Operation of SMH/HMH Series	26	11-1 SMH Series Attachment	49
4-1 Connecting and Disconnecting the Attachment	26	11-2 HMH Series Attachment	51
4-2 Assembling, Mounting and Removing the Cutting Accessory (Optional)	27	11-3 BMH Series Attachment	52
5. Operation of BMH Series	37	11-4 WPD Series Attachment	54
5-1 Connecting and Disconnecting the Attachment	37	12. Warranty	55
5-2 Mounting and Removing the Cutting Accessories (Optional)	37	13. Optional Accessories	56
		14. Disposing of Product	57
		15. Symbols	57

1 User and Intended Purpose / Indication for Each Attachment

1-1 User and Intended Purpose

- User : Qualified Professionals (Surgeon with knowledge and skills required for operating this product).
- Intended Purpose : These devices are designed to be used with the electric or pneumatic surgical instruments, and the cutting accessories manufactured by Nakanishi INC. These devices are intended for: cutting, drilling, removal, and shaping of bones and implants in the fields of Neuro, Spine, Orthopedic, ENT, Oral Maxillofacial, and Plastic and reconstructive surgery.
- Intended patient population : The user shall determine the patient.

1-2 Indication for Each Attachment

Attachment	Application		Surgical fields					
	Cutting bone	Cutting implant	Neuro-surgery	Spine surgery	Orthopedic surgery	ENT surgery	Oral Maxillofacial surgery	Plastic and reconstructive surgery
Slim Handpiece 200	✓		✓	✓	✓	✓	✓	✓
Slim Attachment 300	✓		✓	✓	✓	✓	✓	✓
Standard Handpiece 200	✓		✓	✓	✓	✓	✓	✓
Standard Attachment 300	✓		✓	✓	✓	✓	✓	✓
High Speed Handpiece 200	✓		✓	✓	✓	✓	✓	✓
Primado Drill Attachment	✓		✓	✓	✓	✓	✓	✓
Drill Handpiece 200	✓		✓	✓	✓	✓	✓	✓
Super Slim Handpiece 200	✓			✓				
Perforator Handpiece 200	✓		✓					
Craniotome Handpiece 200	✓		✓					
Contra Angle Handpiece 200	✓			✓			✓	
Bone Saw Attachment 200	✓		✓	✓	✓		✓	✓
Primado Wire Pin Driver Attachment	✓			✓	✓			✓
Metal Cutting Handpiece 200		✓		✓	✓			

2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
⚠ DANGER	Hazard that could result in personal death or serious injury if the safety instructions are not correctly followed.
⚠ WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
⚠ CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

⚠ DANGER

- DO NOT use this product in a room where there is a danger of explosion or in the vicinity of flammable substances. It is especially important not to use this product on a patient to whom a flammable anesthetic agent (dinitrogen monoxide) has been administered.
- Never disassemble or modify the product. Disassembly or modification may cause personal injury or damage to the Motor (including Motor Cord/Motor Hose) or Attachment.

⚠ WARNING

- This is a medical system; do not allow anybody to use it other than surgeons who are familiar with the procedures. Use this system in accordance with its indications for use and proper method of use.
- The Cutting Accessory is for single use only. DO NOT reuse or reprocess it. If you reuse or reprocess it, the following risks may occur:
 - Infection.
 - Personal injury due to damaged Cutting Accessory.
 - System defect due to damaged/deformed Cutting Accessory.
- Irrigate appropriately while cutting in order to prevent necrosis of bones and tissues.
- Perform inspection before use according to "8 Check before Operation". If abnormal vibration or noise is found, the following issues are suspected:
 - The internal consumables (ball bearings, etc.) are worn or damaged.
 - The chuck is open or there is a bad connection of each part.
 - System defect due to damaged/deformed Cutting Accessory.

Using the Attachment in these conditions may lead to heat generation causing thermal injury. Perform "11 Troubleshooting" in this case. If the issue does not improve after troubleshooting, contact your Authorized NSK Dealer.
- DO NOT place the Attachment/Motor (including Motor Cord/Motor Hose) on the patient or the drape covering the patient. Doing so may cause an accident, injury or thermal injury to the patient and/or surgical staff.
- Prevent any debris or other foreign materials from entering into the Attachment. Failure to do so may lead to heat generation, causing thermal injury.
- DO NOT allow strong impact on the product for example, dropping the Attachment/Motor on a hard surface. Doing so may cause personal injury, abnormal heating, fire and/or thermal injury due to product damage.
- After purchase or repair of the Attachment, be sure to clean, lubricate and sterilize the Attachment before use (refer to OM-SZ0911MA Primado2 REPROCESSING MANUAL).
- DO NOT use bent, damaged or deformed Cutting Accessories. If they are used, the Attachment may not clamp the Cutting Accessory securely and it may break or cause accidents.
- DO NOT put excessive pressure on the Cutting Accessory. Doing so may bend or damage the Cutting Accessory, or cause excessive heat generation on the Cutting Accessory, Attachment and Motor, which could lead to thermal injury.
- Avoid continuous use of the product and use it intermittently to prevent excessive heat generation.
In case of heat generation, immediately stop use until the product cools down.
- Be careful not to touch the heated area. Doing so may cause thermal injury.
- When replacing the Cutting Accessory, be sure to slide the Safety Lock on the Hand Switch of the Motor to the OFF position, or completely take your foot off the Foot Control Pedal to stop the Attachment completely. Otherwise, a patient or surgical staff may be injured by unintentional movement of the Motor and the Cutting Accessory.

Precautions for Handling and Operation

⚠ WARNING

- DO NOT operate the Attachment if the markings of the Twist Collet are not aligned (chuck is open) after the Cutting Accessory is mounted. Doing so may cause thermal injury due to heat generation and potentially damage to the Attachment.
- Be sure to adjust the exposure length of the Cutting Accessory outside the operative field and confirm that the exposure length is adjusted as intended before use (when using Slim Attachment 300 or Standard Attachment 300).
- The applicable Attachment for cutting implants is Metal Cutting Handpiece 200. Other Attachments cannot be used.
- DO NOT use the Primado2 Control Unit and Motor in a room with electromagnetic interference or adjacent to portable/mobile RF communication equipment. Use in such environments may affect the Control Unit and Motor. When using equipment generating electromagnetic waves, turn off the power to the Control Unit.

⚠ CAUTION

- When operating the product, always consider the safety of the patient.
- Users shall be responsible for any judgement that relates to the application of this product to a patient.
- Users are responsible for the operational control, maintenance and continual inspection of this System.
- Be sure to read this Operation Manual and the Operation Manuals for "Primado2 Control Unit", "Primado2 Motor", or "Primado Air" in order to become completely familiar with the function of each part before use.
- Be sure to perform "8 Check before Operation" by operating the product with the Motor, Attachments and Cutting Accessory mounted before use. If any abnormal condition is detected, stop using the product and perform "11 Troubleshooting".
- Wear Personal Protective Equipment (PPE) when using the Cutting Accessories.
- Use the Cutting Accessories specified by NSK only. Using other manufacturer's Cutting Accessories may cause a fault or an accident.
- It is recommended that a spare product be kept on hand in case of a breakdown during surgery.
- Never operate the Twist Collet, Fitting/Removal Button, Removal Lever or Removal Ring while operating the Attachment.
- DO NOT touch the rotating part of any Attachments while operating the Attachment.
- If the rotating Cutting Accessory catches hold of operating table drapes, injury and/or damage to the Cutting Accessory and the system may occur. DO NOT allow the rotating Cutting Accessory to come in contact with drapes.
- Make sure that the shank part of the Cutting Accessories to be attached to the Attachment is clean. An unclean shank may cause center deviation or prevent proper clamping.
- After mounting the Cutting Accessory and locking the chuck, pull it lightly to confirm that it is securely connected to the Attachment.
- DO NOT operate the Attachment when the Cutting Accessory is not mounted.
- Before removing the Cutting Accessory from the Attachment, stop the rotation of the Attachment completely.
- Be sure to slide the Safety Lock on the Hand Switch of the Motor to the OFF position when mounting or removing the Attachment (when using the Motor with Hand Switch).
- The Irrigation Nozzle and the Hood/Beak cannot be used at the same time on the Attachment.
- When mounting or removing the Motor and Attachment while using Primado2 Control Unit and Motor, be sure to turn off the Main Power Switch of the Control Unit. If the Main Power Switch is ON, an unintentional movement may occur, which could result in injury to users.
- When mounting or removing the Attachment while using Primado Air Motor, be sure to turn off (0MPa) the gas pressure control valve of the medical gas piping equipment. Mounting or removing the Motor and Attachment when it is ON may cause an unintentional movement, which could result in injury to users (when using PDA-SMH-HS/PDA-BMH-HS).
- While using the "Primado Air Motor: PDA-SMH/PD-BMH (without Hand Switch)", be sure to use "Foot Mode ()". If it is used in "Hand Mode ()", the Motor may continue to rotate and cause an accident. In this case, immediately switch the Control Mode to "Foot Mode ()", or turn off (0MPa) the gas pressure control valve of the medical gas piping equipment to stop the motor rotation.
- Use new Cutting Accessories for every surgical operation for safety.
- This device is for indoor use only.
- Perform regular function and maintenance checks for the devices and parts (see "9 Periodical Maintenance Checks").
- If the product is not used for a long period of time, check if it functions correctly and safely before use.
- If chemicals, solutions or sterilizing liquid is left on the surface of the Attachment, wipe it away immediately. If it is left, it may lead to discoloration, deformation, rust and/or corrosion.
- DO NOT use the following fluids to wipe, immerse or clean the product: strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, and solvents such as benzene or thinner.
- The temperature at the tip of the Attachment may reach up to 60°C depending on the environment.

CAUTION

- Check the correct connection of the following before using Primado2 Control Unit.
 - Connection of the Attachment and Motor.
 - Connection of the Motor Cord Plug and Foot Control Cord Plug to the Control Unit.
 - Connection of the Power Cord Connector to the Control Unit.
 - Connection of the Power Cord Plug to the outlet.
- Check the correct connection of the following before using Primado Air.
 - Connection of the Attachment and Motor.
 - Connection of the Motor Cord Plug and Foot Control Hose Connector to the Foot Control.
 - Connection of the Foot Control Hose (Air Connecting Plug (Schrader Type/DISS Type)) into the medical gas piping equipment (air connecting port).

3 Product Description

3-1 Reference Pages and Package Information by Order Code

One of the Attachments is included in the package.
Each Operation Manual is included in the package.

3-1-1 SMH Series Attachment

_ Slim Handpiece 200*1

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-1A135	P12	P26	P32-33	P47	P49-50	P55-57
P200-1A155						
P200-1A155-C						
P200-1A175						
P200-1A175-C						

*1: One Wrench for Slim/Super Slim (P200-1ARA-W) is included in the package.

_ Super Slim Handpiece 200*1

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-RA330	P12	P26	P32-33	P47	P49-50	P55-57
P200-RA330-L						

*1: One Wrench for Slim/Super Slim (P200-1ARA-W) is included in the package.

_ Standard Handpiece 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-2AMS	P13	P26	P32-33	P47	P49-50	P55-57
P200-2AES						
P200-2AS						
P200-2AM						
P200-2AL						
P200-2SMS						
P200-2SES						
P200-2SS						
P200-2SM						
P200-2SL						

_ Drill Handpiece 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-2SD	P13	P26	P32-33	P47	P49-50	P55-57

_ Metal Cutting Handpiece 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-3MC	P14	P26	P32-33	P47	P49-50	P55-57

_ ContraAngle Attachment 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-CA-4	P14	P26	P34	P47	P49-50	P55-57

_ Perforator Handpiece 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-PER	P14	P26	P36	P47	P49-50	P55-57

_ Craniotome Handpiece 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-CRA	P15	P26	P34	P47	P49-50	P55-57

_ Duraguard 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-CRD-P	P15	-	P35	P47	P49-50	P55-57
P200-CRD-M						
P200-CRD-L						

_ Rotatable Duraguard 200

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-RCRD-P	P15	-	P35	P47	P49-50	P55-57
P200-RCRD-M						
P200-RCRD-L						

Product Description

_ Slim Attachment 300

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P300-1AHA	P16	P26	P27-29	P47	P49-50	P55-57
P300-1AHS						
P300-1T110	P17					
P300-1T110-C						
P300-1T110-PB						
P300-1T130						
P300-1T130-C						
P300-1T130-PB						
P300-1T170						
P300-1T240						

_ Standard Attachment 300

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P300-2A10	P17	P26	P30-31	P47	P49-50	P55-57
P300-2A20						
P300-2A40						
P300-2A60						
P300-2A80						
P300-2S10	P18					
P300-2S20						
P300-2S40						
P300-2S60						
P300-2S80						

3-1-2 HMM Series Attachment**_ High Speed Handpiece 200**

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-3GAS	P18	P26	P32-33	P47	P51-52	P55-57
P200-3GAM						
P200-3GAL						

_ Primado Drill Attachment

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
PD-2SD	P19	P26	P32-33	P47	P51-52	P55-57

_ Duraguard

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
PD-CRD-P	P19	-	P35	P47	P51-52	P55-57
PD-CRD-M						
PD-CRD-L						

3-1-3 BMH Series Attachment**_ Bone Saw Attachment 200**

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product
P200-REC	P20	P37	P38	P47	P52-53	P55-57
P200-SAG			P39			
P200-OSC			P40-41			

Product Description

3-1-4 WPD Series Attachment

_ Primado Wire Pin Driver Attachment

Order Code	Part Names and Specifications	Connecting and Disconnecting the Attachment	Mounting and Removing Cutting Accessories, etc.	Checking	Troubleshooting	Warranty / Optional Accessories Disposing of Product	
PD-DA-W	P21	P42	P43	P47	P54-55	P55-57	
PD-DA-P							
PD-DA-J4*2	P22		P44				
PD-DA-J6*3							
PD-DA-A							
PD-DA-K							
PD-RA-A			P23				P45
PD-RA-M							
PD-RA-H							
PD-RA-T	P24		P46				
PD-SA-R							
PD-SA-S							

*2: One T wrench (PD-TW-J4) is included in the package.

*3: One T wrench (PD-TW-J6) is included in the package.

3-2 Applicable Product

3-2-1 Electric Surgical System

_ Control Unit

Product Name	Order Code	Applicable Attachment
Primado2 Control Unit for 120V	P200-CU-120	All Attachments
Primado2 Control Unit for 230V	P200-CU-230	All Attachments

_ Foot Control

Product Name	Order Code	Applicable Attachment
Primado2 Single Foot Control	FC-73	All Attachments
Primado2 Multi Foot Control	FC-74	

_ Motor

Product Name	Order Code	Applicable Attachment
Primado2 Slim Motor Short	P200-SMH-S	SMH Series Attachment
Primado2 Slim Motor	P200-SMH	
Primado2 Slim Motor/HS	P200-SMH-HS	
Primado2 High Torque Motor	P200-HMH	HMH Series Attachment
Primado2 High Torque Motor/HS	P200-HMH-HS	
Primado2 Micro Bone Saw Motor	P200-BMH	BMH Series Attachment
Primado2 Micro Bone Saw Motor/HS	P200-BMH-HS	
Primado2 Wire Pin Driver	P200-WPD	WPD Series Attachment

3-2-2 Pneumatic Surgical Drilling Machine

_ Foot Control

Product Name	Order Code	Applicable Attachment
Primado Air Foot Control	PDA-FC	SMH Series Attachment BMH Series Attachment

_ Motor

Product Name	Order Code	Applicable Attachment
Primado Air Slim Motor	PDA-SMH	SMH Series Attachment
Primado Air Slim Motor/HS	PDA-SMH-HS	
Primado Air Bone Saw Motor	PDA-BMH	BMH Series Attachment
Primado Air Bone Saw Motor/HS	PDA-BMH-HS	


Product Description

3-3 Part Names and Specifications

3-3-1 SMH Series Attachment


Applicable Motor: P200-SMH-S, P200-SMH, P200-SMH-HS, PDA-SMH, PDA-SMH-HS

Slim Handpiece 200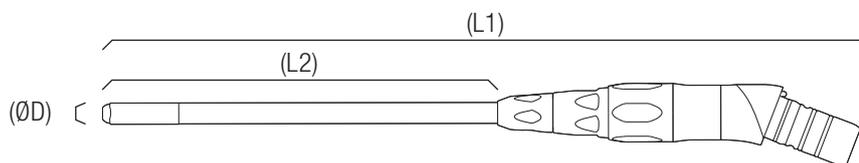

Order Code	P200-1A135	P200-1A155	P200-1A155-C	P200-1A175	P200-1A175-C
Maximum Rotation Speed min ⁻¹	80,000				
Dimension of Attachment (L1) mm	169.7	189.7	189.3	209.7	209.0
Dimension of Bur Guard (ØD x L2) mm	Ø4.8 x 88.6	Ø4.8 x 108.6	Ø4.8 x 108.1	Ø4.8 x 128.6	Ø4.8 x 127.8
Applicable Irrigation Nozzle	-	P200-IN-1A155		P200-IN-1A175	
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.				
CE conformity	CE <small>0197</small>				


Super Slim Handpiece 200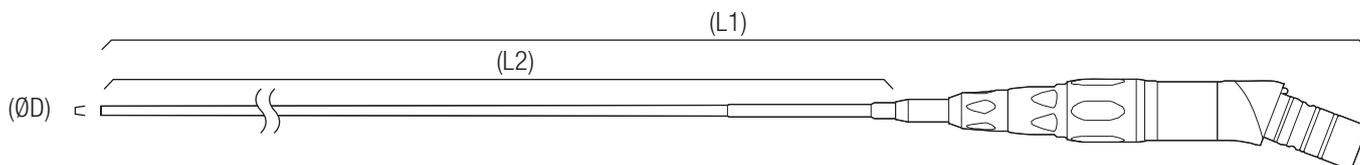

Order Code	P200-RA330	P200-RA330-L
Maximum Rotation Speed min ⁻¹	20,000	
Dimension of Attachment (L1) mm	374.3	374.3
Dimension of Bur Guard (ØD x L2) mm	Ø2.1 x 275.8	Ø3.3 x 275.8
Applicable Irrigation Nozzle	-	
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.	
CE conformity	CE <small>0197</small>	


Standard Handpiece 200

_ Angled Type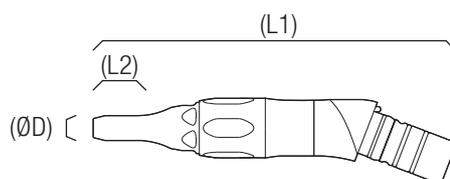


Order Code	P200-2AMS	P200-2AES	P200-2AS	P200-2AM	P200-2AL
Maximum Rotation Speed min ⁻¹	80,000				
Dimension of Attachment (L1) mm	79.7	99.6	119.6	139.6	159.6
Dimension of Bur Guard (ØD x L2) mm	Ø5.5 x 9.7	Ø5.5 x 25.0	Ø5.5 x 45.0	Ø5.5 x 65.0	Ø5.5 x 85.0
Applicable Irrigation Nozzle	PD-IN-D	PD-IN-2ES	PD-IN-2S	PD-IN-2M	PD-IN-2L
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.				
CE conformity	CE ₀₁₉₇				

_ Straight Type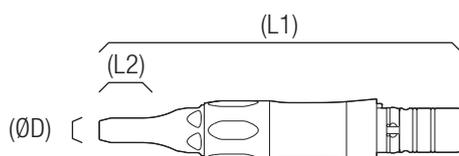

Order Code	P200-2SMS	P200-2SES	P200-2SS	P200-2SM	P200-2SL
Maximum Rotation Speed min ⁻¹	80,000				
Dimension of Attachment (L1) mm	81.1	101.1	121.1	141.1	161.1
Dimension of Bur Guard (ØD x L2) mm	Ø5.5 x 9.7	Ø5.5 x 25.0	Ø5.5 x 45.0	Ø5.5 x 65.0	Ø5.5 x 85.0
Applicable Irrigation Nozzle	PD-IN-D	PD-IN-2ES	PD-IN-2S	PD-IN-2M	PD-IN-2L
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.				
CE conformity	CE ₀₁₉₇				


Drill Handpiece 200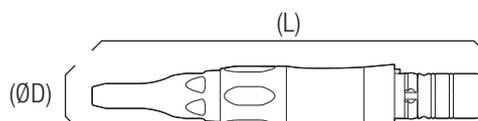

Order Code	P200-2SD
Maximum Rotation Speed min ⁻¹	80,000
Dimension of Attachment (ØD x L) mm	Ø13.2 x 86.0
Applicable Irrigation Nozzle	PD-IN-D
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	CE ₀₁₉₇


Product Description

Metal Cutting Handpiece 200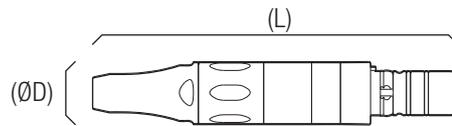


Order Code	P200-3MC
Maximum Rotation Speed min ⁻¹	80,000
Dimension of Attachment (ØD x L) mm	Ø14.0 x 81.4mm
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	

ContraAngle Attachment 200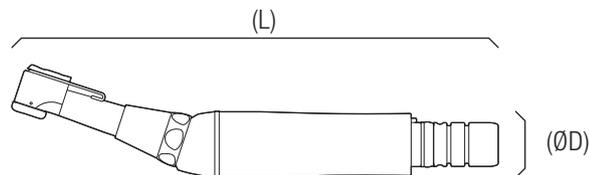


Order Code	P200-CA-4
Maximum Rotation Speed min ⁻¹	20,000
Gear Ratio	4:1
Dimension of Attachment (ØD x L) mm	Ø14.8 x 109.0
Applicable Cutting Accessories	Commercially available Ø2.35mm contra burs
CE conformity	

Perforator Handpiece 200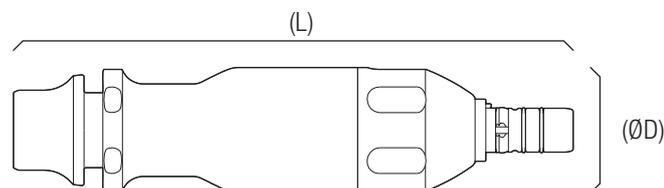


Order Code	P200-PER
Maximum Rotation Speed min ⁻¹	1,250
Dimension of Attachment (ØD x L) mm	Ø28.2 x L125.0
Chuck	Commercially available Perforator with Hudson end
CE conformity	

Craniotome Handpiece 200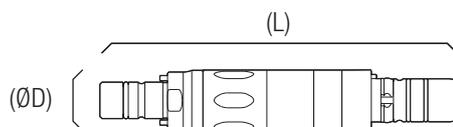


Order Code	P200-CRA		
Maximum Rotation Speed min ⁻¹	80,000		
Dimension of Attachment (ØD x L) mm	Ø14.0 x L81.4		
Applicable Duraguard	P200-CRD-P P200-RCRD-P	P200-CRD-M P200-RCRD-M	P200-CRD-L P200-RCRD-L
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.		
CE conformity			

Duraguard 200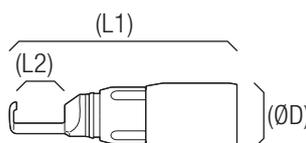

Order Code	P200-CRD-P	P200-CRD-M	P200-CRD-L
Applicable Attachment	P200-CRA		
Dimension (ØD x L1) mm	Ø15.4 x L54.3	Ø15.4 x L58.7	Ø15.4 x L68.5
Length of Guard (L2) mm	11.35	15.35	25.35
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.		
CE conformity			

Rotatable Duraguard 200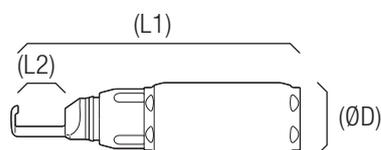


Order Code	P200-RCRD-P	P200-RCRD-M	P200-RCRD-L
Applicable Attachment	P200-CRA		
Dimension (ØD x L1) mm	Ø17.5 x L69.7	Ø17.5 x L73.7	Ø17.5 x L83.7
Length of Guard (L2) mm	11.35	15.35	25.35
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.		
CE conformity			

Product Description


Slim Attachment 300

_ Angled Type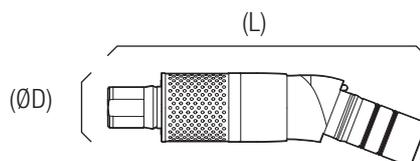


Order Code	P300-1AHA			
Maximum Rotation Speed min ⁻¹	80,000			
Exposure range of Cutting Accessories (Variable)	0 - 10			
Dimension (ØD x L) mm	Ø14.5 x 67.0			
Applicable Tube	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170	P300-1T240
Applicable Cutting Accessories	Refer to OM-SZ0958MA CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE.			
CE conformity	CE $\frac{2011}{65}$			

_ Straight Type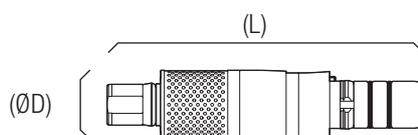

Order Code	P300-1AHS			
Maximum Rotation Speed min ⁻¹	80,000			
Exposure range of Cutting Accessories (Variable)	0 - 10			
Dimension (ØD x L) mm	Ø14.5 x 66.0			
Applicable Tube	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170	P300-1T240
Applicable Cutting Accessories	Refer to OM-SZ0958MA CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE.			
CE conformity	CE $\frac{2011}{65}$			


_ Slim Tube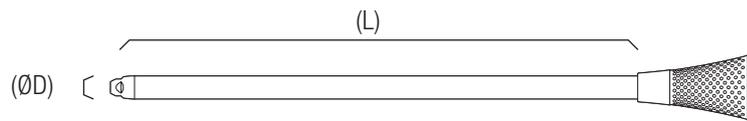

Order Code	P300-1T110 P300-1T110-C P300-1T110-PB	P300-1T130 P300-1T130-C P300-1T130-PB	P300-1T170	P300-1T240
Applicable Slim Attachment Hub	P300-1AHS/P300-1AHA			
Dimension of Bur Guard (ØD x L)	Ø4.8 x 110.0	Ø4.8 x 130.0	Ø4.8 x 170.0	Ø4.8 x 240.0
Applicable Hood / Beak	P300-1T-H P300-1T-B			
Applicable Irrigation Nozzle	P200-IN-1A155	P200-IN-1A175	-	
CE conformity				

Standard Attachment 300


_ Angled Type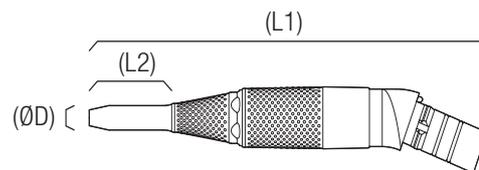

Order Code	P300-2A10	P300-2A20	P300-2A40	P300-2A60	P300-2A80
Maximum Rotation Speed min ⁻¹	80,000				
Exposure range of Cutting Accessories (Variable)	0 - 12.5				
Dimension of Attachment (L1) mm	88.0	103.0	123.0	143.0	163.0
Dimension of Bur Guard (ØD x L2) mm	Ø5.5 x 10.0	Ø5.5 x 20.0	Ø5.5 x 40.0	Ø5.5 x 60.0	Ø5.5 x 80.0
Applicable Beak	-		P300-2-B		
Applicable Irrigation Nozzle	PD-IN-D		PD-IN-2ES	PD-IN-2S	PD-IN-2M
Applicable Cutting Accessories	Refer to OM-SZ0958MA CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE.				
CE conformity					

Product Description

_ Straight Type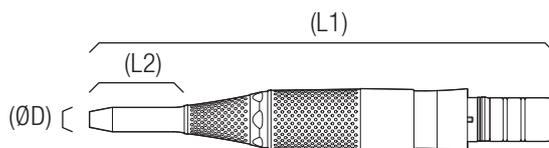


Order Code	P300-2S10	P300-2S20	P300-2S40	P300-2S60	P300-2S80
Maximum Rotation Speed min ⁻¹	80,000				
Exposure range of Cutting Accessories (Variable)	0 - 12.5				
Dimension of Attachment (L1) mm	89.0	104.0	124.0	144.0	164.0
Dimension of Bur Guard (ØD x L2) mm	Ø5.5 x 10.0	Ø5.5 x 20.0	Ø5.5 x 40.0	Ø5.5 x 60.0	Ø5.5 x 80.0
Applicable Beak	-		P300-2-B		
Applicable Irrigation Nozzle	PD-IN-D		PD-IN-2ES	PD-IN-2S	PD-IN-2M
Applicable Cutting Accessories	Refer to OM-SZ0958MA CUTTING ACCESSORIES - P300 ATTACHMENT CORRESPONDENCE TABLE.				
CE conformity	CE $\frac{S}{M}$				

3-3-2 HMH Series Attachment


Applicable Motor: P200-HMH, P200-HMH-HS

High Speed Handpiece 200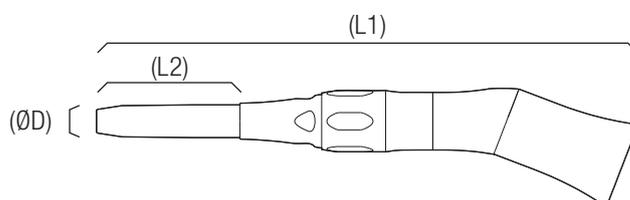


Order Code	P200-3GAS	P200-3GAM	P200-3GAL
Maximum Rotation Speed min ⁻¹	100,000		
Dimension of Attachment (L1) mm	122.7	142.7	122.4
Dimension of Bur Guard (ØD x L2) mm	Ø7.5 x 32.6	Ø7.5 x 52.6	Ø7.5 x 92.6
Applicable Irrigation Nozzle	PD-IN-3S	PD-IN-3M	PD-IN-3L
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.		
CE conformity	CE $\frac{S}{M}$		

Primado Drill Attachment _ Straight Type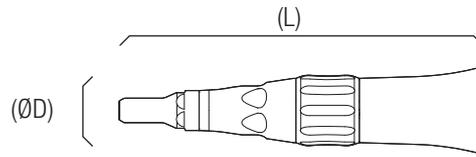

Order Code	PD-2SD
Maximum Rotation Speed min ⁻¹	60,000
Dimension of Attachment (ØD x L) mm	Ø15.9 x 81.9
Applicable Irrigation Nozzle	PD-IN-D
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	

Duraguard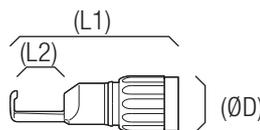

Order Code	PD-CRD-P	PD-CRD-M	PD-CRD-L
Applicable Attachment	PD-CRA		
Dimension (ØD x L1) mm	Ø13.8 x 40.6	Ø13.8 x 44.6	Ø13.8 x 54.6
Length of Guard (L2) mm	11.3	15.3	25.3
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.		
CE conformity			


Product Description

3-3-3 P200-BMH Series Attachment

Applicable Motor: P200-BMH, P200-BMH-HS


Bone Saw Attachment 200

_ Primado2 Rec. Attachment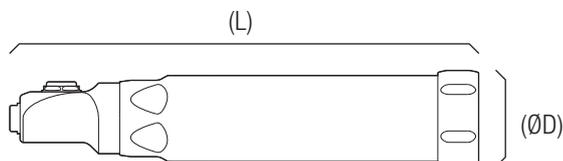


Order Code	P200-REC
Maximum Rotation Speed min ⁻¹	20,000
Dimension of Attachment (ØD x L) mm	Ø22 x 105.1
Applicable Irrigation Nozzle	P200-IN-BS
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	

_ Primado2 Sag. Attachment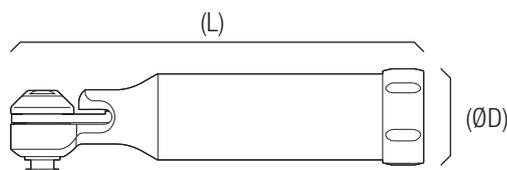

Order Code	P200-SAG
Maximum Rotation Speed min ⁻¹	20,000
Dimension of Attachment (ØD x L) mm	Ø22 x 92.3
Applicable Irrigation Nozzle	P200-IN-BS
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	

_ Primado2 Osc. Attachment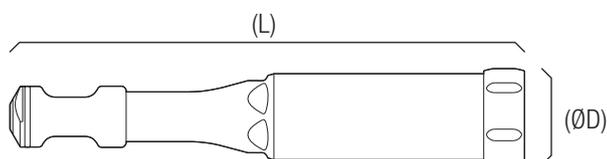


Order Code	P200-OSC
Maximum Rotation Speed min ⁻¹	20,000
Dimension of Attachment (ØD x L) mm	Ø22 x 115.4
Applicable Adaptor	PD-OSC-A
Applicable Irrigation Nozzle	P200-IN-BS
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	

3-3-4 WPD Series Attachment

Applicable Motor: P200-WPD


Primado Wire Pin Driver Attachment

_ Primado Wire Collet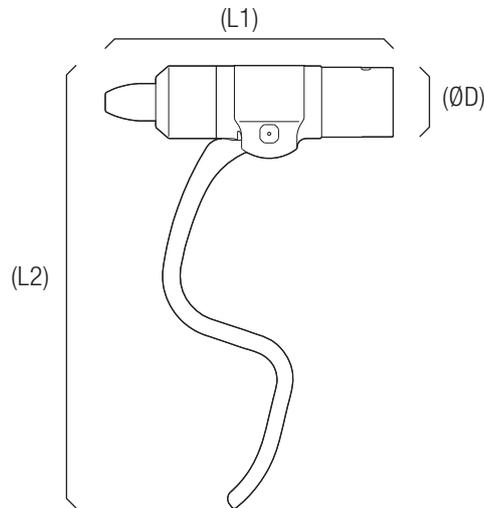

Order Code	PD-DA-W
Maximum Rotation Speed min ⁻¹	1,200
Chuck Capacity mm	Ø0.6 - 1.9
Dimension of Attachment (ØD x L1 x L2) mm	Ø23 x 91 x 148
Applicable Cutting Accessories	Commercially available Ø0.6 - Ø1.9mm wire or pin
CE conformity	


_ Primado Pin Collet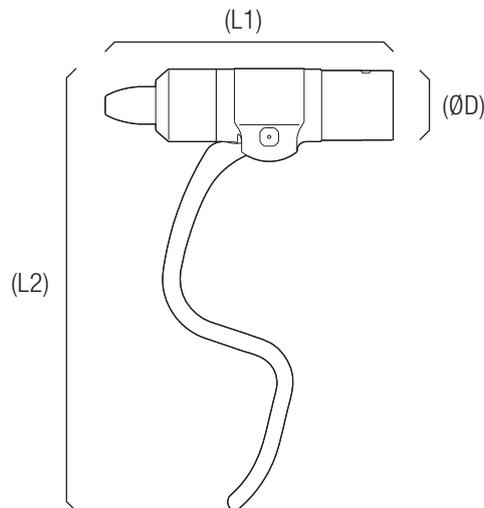

Order Code	PD-DA-P
Maximum Rotation Speed min ⁻¹	1,200
Chuck Capacity mm	Ø2.0 - 3.2
Dimension of Attachment (ØD x L1 x L2) mm	Ø23 x 91 x 148
Applicable Cutting Accessories	Commercially available Ø2.0 - Ø3.2mm wire or pin
CE conformity	


Product Description

_ Jacobs Keyed Drill Chuck 4.0mm/6.5mm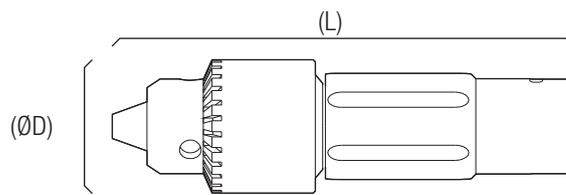


Order Code	PD-DA-J4	PD-DA-J6
Maximum Rotation Speed min ⁻¹	1,200	
Chuck Capacity mm	Ø0.5 - 4.0	Ø0.5 - 6.5
Cannulation mm	Ø4.2	
Dimension of Attachment (ØD x L) mm	Ø21 x 93	Ø31 x 108
Applicable Cutting Accessories	Commercially available drill bit, wire or pin with Ø4.0mm or less shank	Commercially available drill bit, wire or pin with Ø6.5mm or less shank
CE conformity	CE _{2011/65}	

_ AO Drill Chuck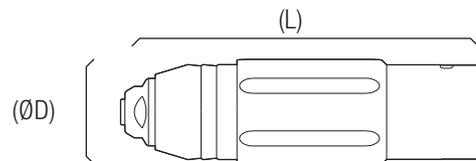


Order Code	PD-DA-A	
Maximum Rotation Speed min ⁻¹	1,200	
Cannulation mm	Ø1.5	
Dimension of Attachment (ØD x L) mm	Ø25 x 84	
Applicable Cutting Accessories	Commercially available AO-type drill bit	
CE conformity	CE _{2011/65}	

_ Keyless Drill Chuck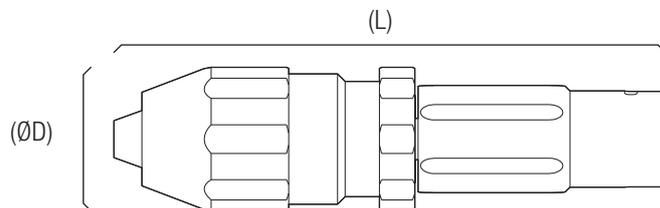


Order Code	PD-DA-K	
Maximum Rotation Speed min ⁻¹	1,200	
Chuck Capacity mm	Ø0.6 - 7.4	
Cannulation mm	Ø4.2	
Dimension of Attachment (ØD x L) mm	Ø31.5 x 84	
Applicable Cutting Accessories	Commercially available drill bit, wire or pin with Ø7.4mm or less shank	
CE conformity	CE _{2011/65}	

_ AO Reamer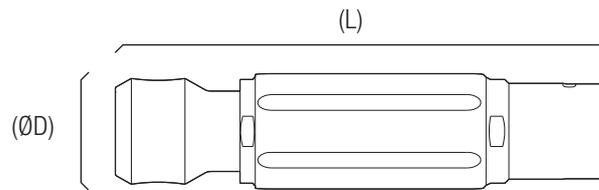


Order Code	PD-RA-A
Maximum Rotation Speed min ⁻¹	300
Cannulation mm	Ø4.05
Dimension of Attachment (ØD x L) mm	Ø27 x 113
Applicable Cutting Accessories	Commercially available reamer bit with AO end
CE conformity	CE ₀₁₉₇

_ Hudson Modified Trinkle Reamer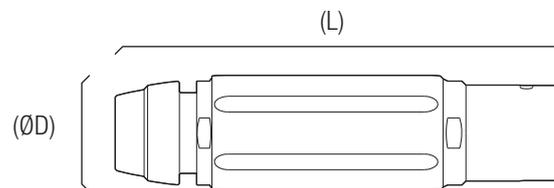

Order Code	PD-RA-M
Maximum Rotation Speed min ⁻¹	300
Cannulation mm	Ø4.05
Dimension of Attachment (ØD x L) mm	Ø27 x 107
Applicable Cutting Accessories	Commercially available reamer bit with Hudson modified trinkle end
CE conformity	CE ₀₁₉₇


_ Hudson Reamer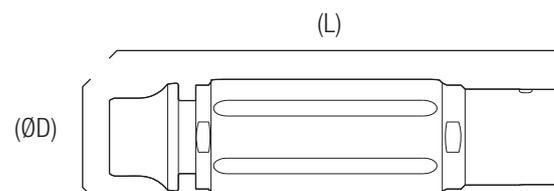

Order Code	PD-RA-H
Maximum Rotation Speed min ⁻¹	300
Cannulation mm	Ø4.05
Dimension of Attachment (ØD x L) mm	Ø27 x 104
Applicable Cutting Accessories	Commercially available reamer bit with Hudson end
CE conformity	CE ₀₁₉₇


Product Description

_ Trinkle Reamer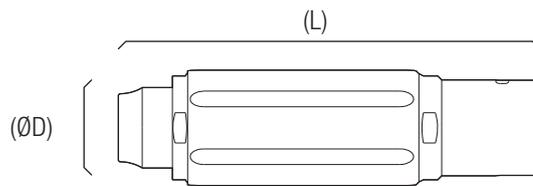


Order Code	PD-RA-T
Maximum Rotation Speed min ⁻¹	300
Cannulation mm	Ø4.05
Dimension of Attachment (ØD x L) mm	Ø27 x 97
Applicable Cutting Accessories	Commercially available reamer bit with trinkle end
CE conformity	CE ₀₁₉₇

_ Reciprocating Saw Attachment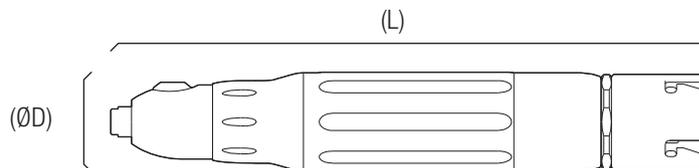


Order Code	PD-SA-R
Maximum Rotation Speed min ⁻¹	12,000
Dimension of Attachment (ØD x L) mm	Ø23.5 x 138
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	CE ₀₁₉₇

_ Sagittal Saw Attachment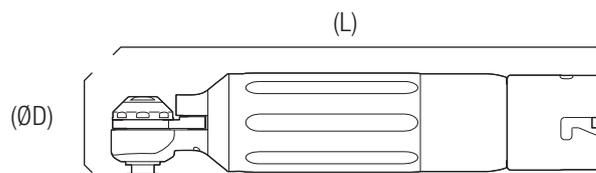

Order Code	PD-SA-S
Maximum Rotation Speed min ⁻¹	12,000
Dimension of Attachment (ØD x L) mm	Ø23.5 x 114
Applicable Cutting Accessories	Refer to OM-SZ0920MA CUTTING ACCESSORIES - ATTACHMENT CORRESPONDENCE TABLE.
CE conformity	CE ₀₁₉₇

Use, Storage and Transportation Environment

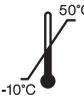Use environment			-
Store and transportation environment			

4 Operation of SMH/HMH Series

4-1 Connecting and Disconnecting the Attachment


⚠ CAUTION

- Be sure to slide the Safety Lock on the Hand Switch of the Motor to the OFF position when connecting and disconnecting the Attachment (when using the Motor with Hand Switch).
- Refer to the Operation Manual of the Motor.
- Remove the Cutting Accessory before connecting and disconnecting the Attachment to prevent personal injury.
- After connecting the Attachment, make sure that it is correctly connected to the Motor.

4-1-1 SMH Series Attachment

_Connecting

- 1 Insert the Attachment (A) straight into the Motor (B).
- 2 Align the '▶' mark of the Attachment (C) and '●' mark of the Motor (D), then push the Attachment straight in until it clicks to connect firmly.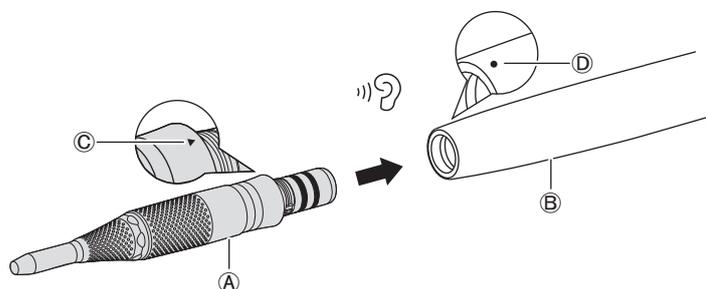


_Disconnecting

Pull the Attachment (A) straight out from the Motor (B).

4-1-2 HMH Series Attachment

_Connecting

- 1 Insert the Attachment (A) straight into the Motor (B).
- 2 Rotate the Attachment (A) rightward or leftward so that the Alignment Pin (C) matches, then push the Attachment in until it clicks to connect firmly.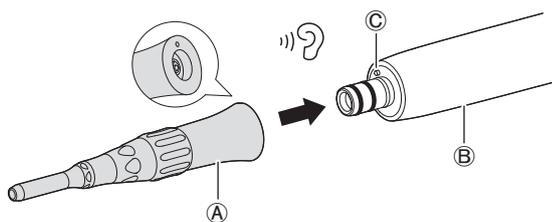

_Disconnecting

Pull the Attachment (A) straight out from the Motor (B).

4-2 Assembling, Mounting and Removing the Cutting Accessory (Optional)

▲ WARNING

- Be sure to use the NSK recommended Cutting Accessory for each Attachment (see "3-3 Part Names and Specifications"). When using an unrecommended product, it cannot be firmly fixed. This may lead to the product coming off during the surgical operation.
- DO NOT rotate the Attachment if the chuck of the Attachment is open. Doing so may cause thermal injury and/or damage to the product due to heat generation of the Attachment.

▲ CAUTION

- After mounting the Cutting Accessory and locking the chuck, pull it lightly to confirm that it is securely connected to the Attachment.
- Before use, confirm that the Cutting Accessory does not come off by operating the Attachment.
- DO NOT operate the Attachment when the Cutting Accessory is not mounted.
- Before mounting or removing the Cutting Accessory, be sure to stop the Attachment completely.
- For Perforator Handpiece 200 or Perforator Handpiece, use a commercially available perforator with Hudson end. Use the perforator in accordance with its Operation Manual.
- Mount the Cutting Accessory to the Attachment after mounting the Irrigation Nozzle.
- Be sure to use the NSK-specified Irrigation Nozzle (see "3-3 Part Names and Specifications"). For the related connection on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.

4-2-1 Slim Attachment 300


Connecting and Disconnecting the Slim Tube

▲ CAUTION

- Be sure to remove the Cutting Accessories from the Attachment before disconnecting the Tube.

Connecting

Insert the Tube straight into the Hub while rotating it slightly, then push the Tube in until it clicks.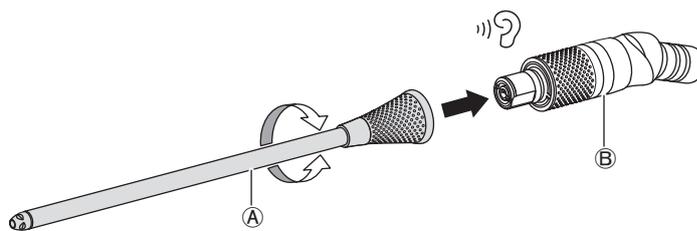

Disconnecting


Pull the Tube (A) straight out from the Hub (B).

Operation of SMH/HMH Series

Mounting the Irrigation Nozzle (Optional)

Mounting

- 1 Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- 2 Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).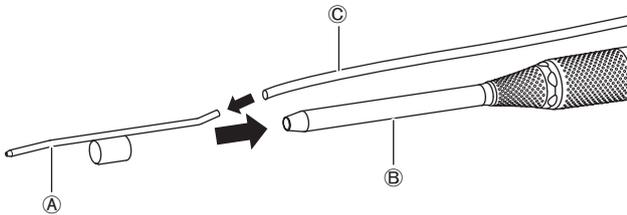

* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.


Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


Mounting and Removing the Cutting Accessories (Optional)

Mounting


- 1 Rotate the Twist Collet (A) in the direction of (B) until it clicks. Insert the Cutting Accessory (C) all the way while twisting it.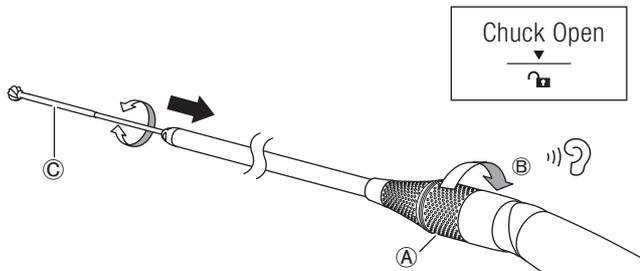


- 2 Push or pull the Cutting Accessory to adjust the exposure length as required (when using Slim Attachment 300).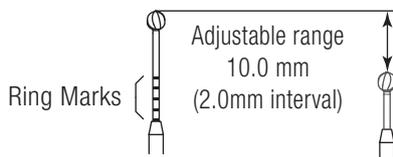

- 3 Rotate the Twist Collet (A) in the direction of (D) until it clicks.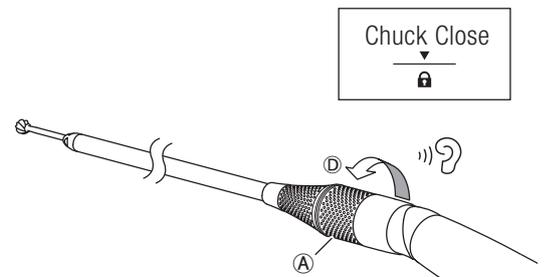

- 4 Pull the Cutting Accessory (C) lightly to confirm that it is securely connected to the Attachment.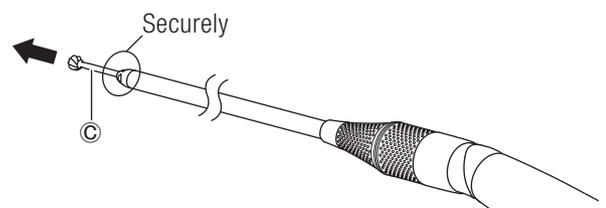

Removing

Perform steps 1 through 3 of the mounting procedure in the reverse order.

NOTICE


- For Slim Attachment 300 applicable Cutting Accessory, the exposure length of its tip can be adjusted within the chucking range by changing the chucking position.
- The exposure variable range of the Cutting Accessory is 10mm.

Ring Marks

Some of the NSK Cutting Accessories have Ring Marks.

Check the Ring Marks of each Cutting Accessory to mount it appropriately.

* There is no Ring Mark on the Cutting Accessories not shown in this Operation Manual.

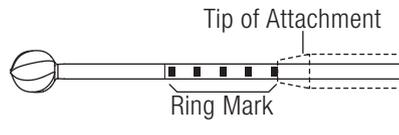Applicable Attachment	Applicable Tube	Applicable Cutting Accessories	Ring Mark	Correct Mounting Position
P300-1AHS P300-1AHA	P300-1T110 P300-1T110-C P300-1T110-PB	PDS-1□□110-□□		 <p>There are 6 Ring Marks. Each Ring Mark goes inside the Attachment each time the exposure length is adjusted.</p>
	P300-1T130 P300-1T130-C P300-1T130-PB	PDS-1□□130-□□		
	P300-1T170	PDS-1□□170-□□		
	P300-1T240	PDS-1□□240-□□		

Mounting and Removing the Hood/Beak (Optional)

⚠ CAUTION


- Be sure to mount the Hood/Beak after the Cutting Accessory is mounted on the Attachment.
- Insert the Hood/Beak until '▶' mark on the Bur Guard is hidden when mounting the Hood/Beak.
- Use the Cutting Accessory with its tip of $\varnothing 3\text{mm}$ or less when mounting the Hood/Beak.
*The Hood/Beak cannot be used for attachments other than Slim Attachment 300 or Standard Attachment 300.
- Be sure to completely stop rotation of the Attachment before mounting/removing the Hood/Beak.

Mounting

- 1 Make sure that the rotation of the Attachment is stopped.
- 2 Mount the Hood (A)/Beak (B) on the Bur Guard (C).

Removing

- 1 Make sure that the rotation of the Attachment is stopped.
- 2 Pull the Hood (A)/Beak (B) straight out from the Bur Guard (C).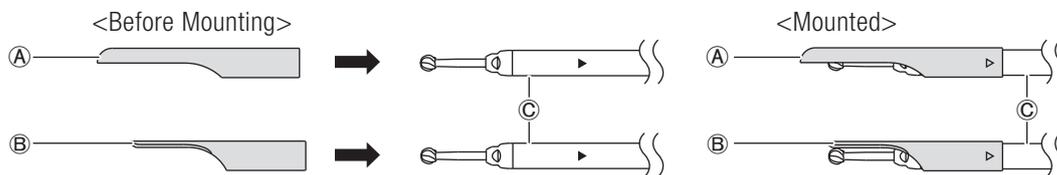


Operation of SMH/HMH Series

4-2-2 Standard Attachment 300

Mounting the Irrigation Nozzle (Optional)

_Mounting

- 1 Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- 2 Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).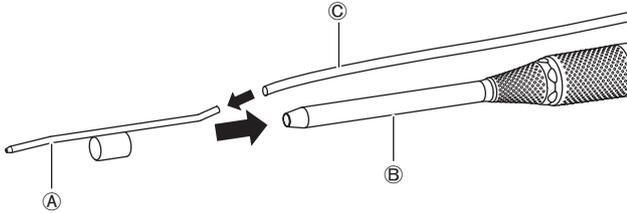


* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.

Mounting and Removing the Cutting Accessories (Optional)



_Mounting

- 1 Pull the Twist Collet (A) in the direction of (B) while rotating it in the direction of (C) until it clicks. Insert the Cutting Accessory (D) all the way while twisting it slightly.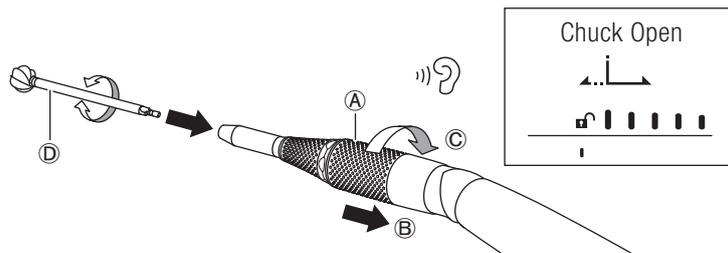


- 2 Rotate the Twist Collet in the direction of (E) to fix the Cutting Accessory.

- 3 Pull the Cutting Accessory (D) lightly to confirm that it is securely connected to the Attachment.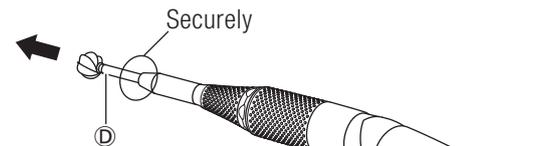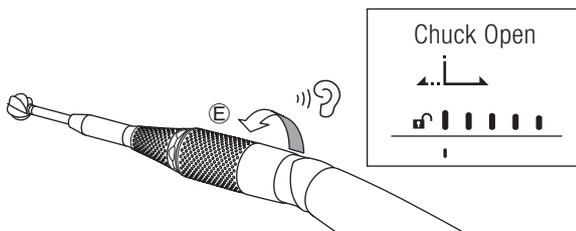

_Adjusting the Exposure Length of the Cutting Accessory (when using Standard Attachment 300)

The exposure length of the Cutting Accessory can be adjusted by rotating the Twist Collet of the Attachment (B) in the direction of (C) or (E) while pulling it lightly in the direction of (B). Refer to the figure below.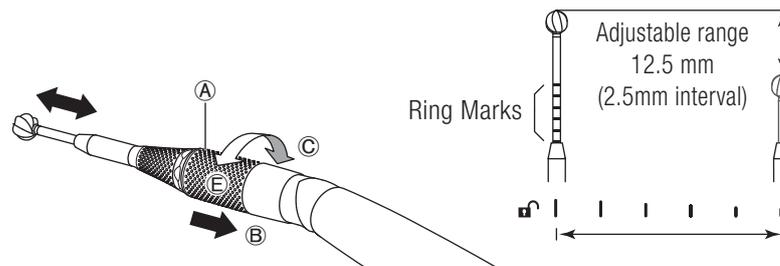

_Removing



Perform steps 1 and 2 of the mounting procedure in the reverse order.

Ring Marks

Some of the NSK Cutting Accessories have Ring Marks.

Check the Ring Marks of each Cutting Accessory to mount it appropriately.

* There is no Ring Mark on the Cutting Accessories not shown in this Operation Manual.

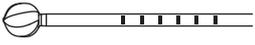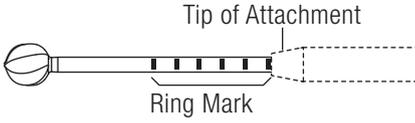Applicable Attachment	Applicable Cutting Accessories	Ring Mark	Correct Mounting Position
P300-2S10 P300-2A10 P300-2S20 P300-2A20 P300-2S40 P300-2A40 P300-2S60 P300-2A60 P300-2S80 P300-2A80	PDS-2□□UL-□□		 <p>There are 6 Ring Marks. Each Ring Mark goes inside the Attachment each time the exposure length is adjusted.</p>

Mounting and Removing the Beak (Optional)

⚠ CAUTION


- Be sure to mount the Beak after the Cutting Accessory is mounted on the Attachment.
- Insert the Beak until '▶' mark on the Bur Guard is hidden when mounting the Beak.
- Use the Cutting Accessory with its tip of $\varnothing 4\text{mm}$ or less when mounting the Beak.
*The Beak cannot be used for attachments other than Slim Attachment 300 or Standard Attachment 300.
- Be sure to completely stop rotation of the Attachment before mounting/removing the Beak.

— Mounting

- 1 Make sure that the rotation of the Attachment is stopped.
- 2 Mount the Beak (A) on the Bur Guard (B).

— Removing

- 1 Make sure that the rotation of the Attachment is stopped.
- 2 Pull the Beak (A) straight out from the Bur Guard (B).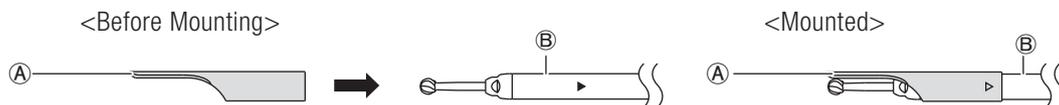


**4-2-3 Slim Handpiece 200 / Super Slim Handpiece 200 /
Standard Handpiece 200 / Drill Handpiece 200 /
Metal Cutting Handpiece 200 / High Speed Handpiece 200 /
Primado Drill Attachment**

Mounting the Irrigation Nozzle (Optional)

— Mounting

- ① Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- ② Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).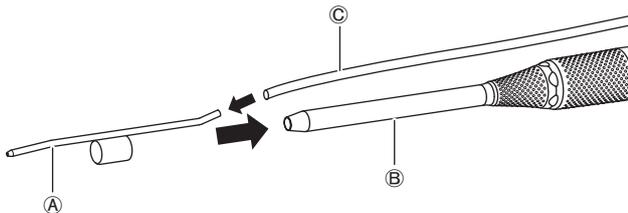

— Removing



Perform steps 1 and 2 of the mounting procedure in the reverse order.

* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.


Mounting and Removing the Cutting Accessories (Optional)

— Mounting

- ① Rotate the Twist Collet (A) in the direction of (B) until it clicks. Insert the Cutting Accessory (C) all the way while twisting it.
- ③ Pull the Cutting Accessory (C) lightly to confirm that it is securely connected to the Attachment.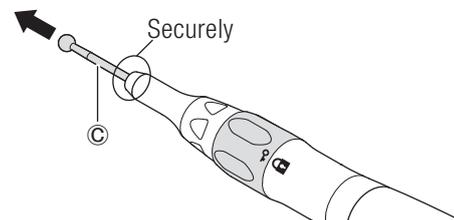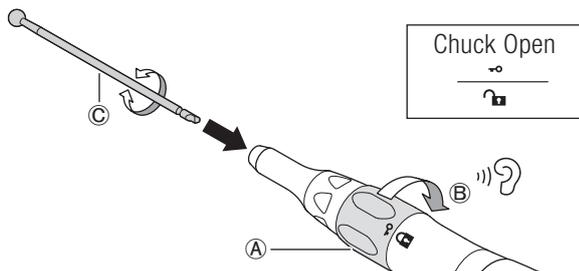

- ② Rotate the Twist Collet (A) in the direction of (D) until it clicks.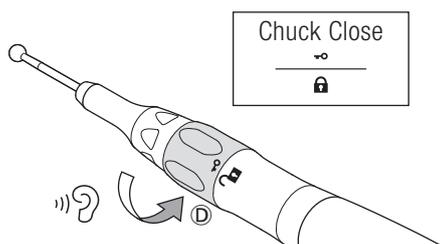

— Removing







Perform steps 1 and 2 of the mounting procedure in the reverse order.

Ring Marks

Some of the NSK Cutting Accessories have Ring Marks.

Check the Ring Marks of each Cutting Accessory to mount it appropriately.

* There is no Ring Mark on the Cutting Accessories not shown in this Operation Manual.

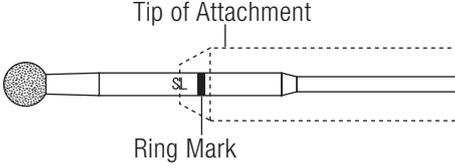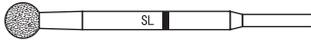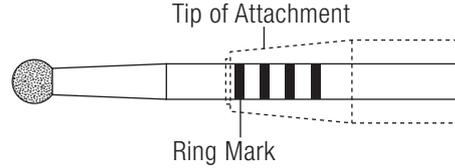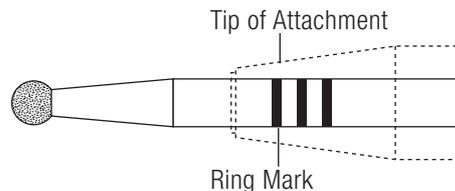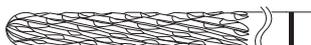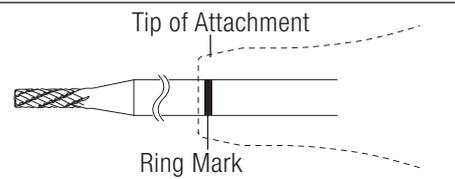Applicable Attachment	Applicable Cutting Accessories	Ring Mark	Correct Mounting Position
P200-1A135	PDS-1□□135-□□		 <p>Tip of Attachment</p> <p>Ring Mark</p> <p>The identification letter should be partially visible. The Ring Mark should not be visible, as it is hidden inside the Attachment.</p>
P200-1A155 P200-1A155-C	PDS-1□□155-□□		
P200-1A175 P200-1A175-C	PDS-1□□175-□□		
P200-RA330 P200-RA330-L	PDS-1□□330-□□	No Ring Mark	
P200-2SMS P200-2AMS	PDS-2□□MS-□□		 <p>Tip of Attachment</p> <p>Ring Mark</p> <p>There are 4 Ring Marks. All Ring Marks should not be visible, as they are hidden inside the Attachment.</p>
P200-2SES P200-2AES	PDS-2□□ES-□□		
P200-2SS P200-2AS	PDS-2□□S-□□		
P200-2SM P200-2AM	PDS-2□□M-□□		
P200-2SL P200-2AL	PDS-2□□L-□□		
P200-3GAS	PDS-3□□S-□□		 <p>Tip of Attachment</p> <p>Ring Mark</p> <p>The Ring Marks should not be visible, as they are hidden inside the Attachment.</p>
P200-3GAM	PDS-3□□M-□□		
P200-3GAL	PDS-3□□L-□□		
P200-3MC	PDS-3MCS-□□		 <p>Tip of Attachment</p> <p>Ring Mark</p>


Operation of SMH/HMH Series

4-2-4 ContraAngle Attachment 200


Mounting and Removing the Cutting Accessories (Commercially Available)

_ Mounting


1 Slide the Latch (A) in the direction of (B).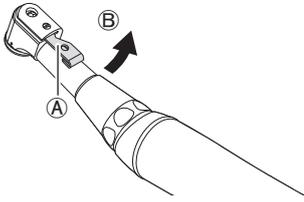


2 Match the notch of the bur (C) to the corresponding part of the shaft and push the bur all the way through.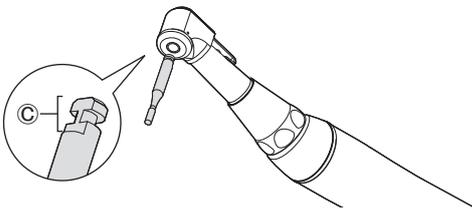

3 Slide the Latch (A) in the direction of (D) to set the bur in place.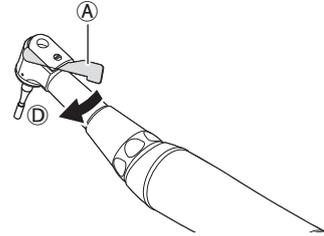

4 After mounting the bur (E), pull it lightly to confirm that it is securely connected to the Attachment.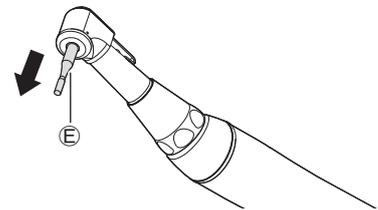

_ Removing


Perform steps 1 through 3 of the mounting procedure in the reverse order.

4-2-5 Craniotome Handpiece 200, Craniotome Handpiece


Mounting and Removing the Cutting Accessories (Optional)

_ Mounting


1 Rotate the Twist Collet (A) in the direction of (B) until it clicks. Insert the Cutting Accessory (C) all the way while twisting it.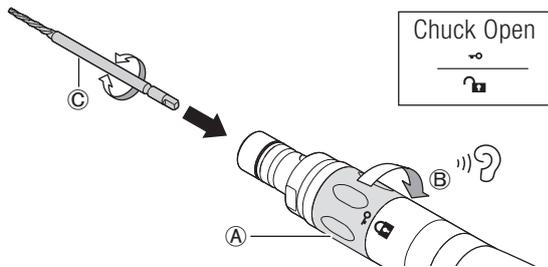

3 Pull the Cutting Accessory (C) lightly to confirm that it is securely connected to the Attachment.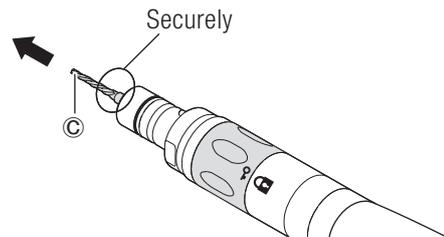

2 Rotate the Twist Collet (A) in the direction of (D) until it clicks.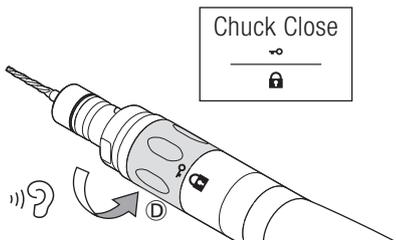

_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.



Mounting and Removing the Duraguard

Applicable Attachment	Applicable Duraguard	
	Duraguard	Rotatable Duraguard
P200-CRA	P200-CRD-P P200-CRD-M P200-CRD-L	P200-RCRD-P P200-RCRD-M P200-RCRD-L
PD-CRA	PD-CRD-P PD-CRD-M PD-CRD-L	-

Craniotome Handpiece 200

_ Mounting

- 1 Insert the Duraguard (A) while twisting it through the tip of the Attachment (B).
- 2 Rotate the Removal Ring (C) in the direction of (D) to set the Duraguard (A) in place.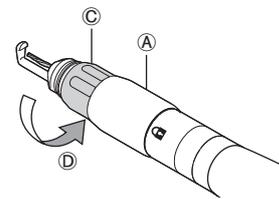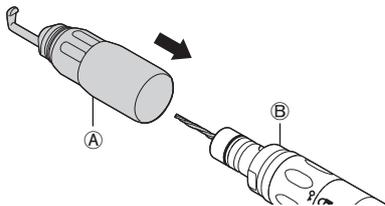



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.

Craniotome Handpiece*

_ Mounting

- 1 Insert the Duraguard (A) while twisting it through the tip of the Attachment (B).
- 2 Rotate the Removal Ring (C) in the direction of (D) to set the Duraguard (A) in place.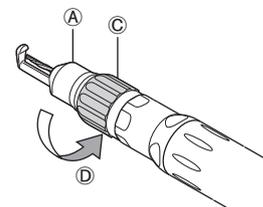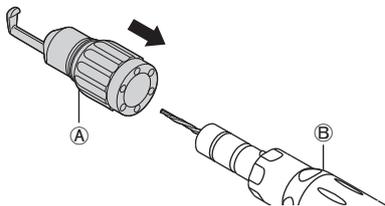

_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.



*PD-CRA is no longer available.

4-2-6 Perforator Handpiece 200


Mounting and Removing the Perforator (Commercially Available)

_ Mounting

- 1 Align the shank of the perforator with Hudson type chuck (B) with the groove of the Attachment (C) while pulling the Removal Lever (A), and push it all the way.
- 2 Release the Removal Lever (A) to set the perforator in place.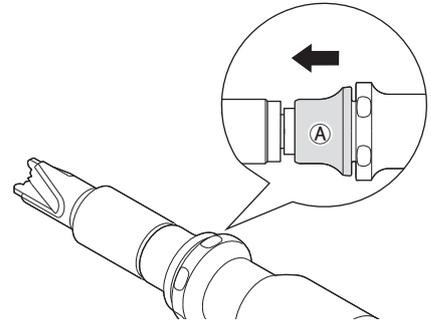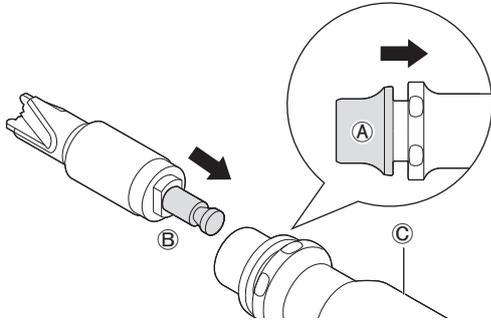

- 3 After mounting the perforator (D), pull it lightly to confirm that it is securely connected to the Attachment.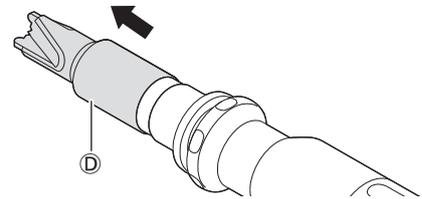

_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.

5 Operation of BMH Series


5-1 Connecting and Disconnecting the Attachment

⚠ CAUTION

- Be sure to slide the Safety Lock on the Hand Switch of the Motor to the OFF position when connecting and disconnecting the Attachment (when using the Motor with Hand Switch).
- Refer to the Operation Manual of the Motor.
- Remove the Cutting Accessory before connecting and disconnecting the Attachment to prevent personal injury.
- After connecting the Attachment, make sure that it is correctly connected to the Motor.

Connecting

- 1 Insert the Attachment (A) straight into the Motor (B).
- 2 Rotate the Attachment (A) so that the Alignment Pin (C) and Alignment Hole (D) matches, then push the Attachment in until it clicks to connect firmly. (There are 4 Alignment Holes.)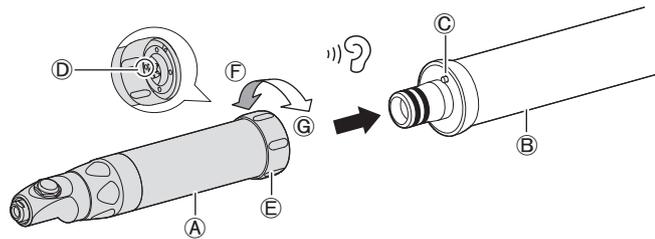

- 3 Tighten the Removal Ring of the Attachment (E) manually in the direction of (G) to fix it firmly.

Disconnecting

Loosen the Removal Ring (E) in the direction of (F) and pull the Attachment (A) straight out from the Motor (B).

5-2 Mounting and Removing the Cutting Accessories (Optional)

⚠ WARNING

- Be sure to use a NSK recommended Cutting Accessory or Adaptor for Oscillating Intra Oral Blade for each Attachment (see "3-3 Part Names and Specifications").
When using an unrecommended product, it cannot be firmly fixed. This may lead to the product coming off during the surgical operation.
- DO NOT rotate the Attachment if the chuck of the Attachment is open. Doing so may cause thermal injury and/or damage to the product due to heat generation of the Attachment.

⚠ CAUTION

- After mounting the Cutting Accessory or Adaptor for Oscillating Intra Oral Blade, pull it lightly to confirm that Cutting Accessory or Adaptor for Oscillating Intra Oral Blade is securely connected to the Attachment.
- Before use, confirm that the Cutting Accessory or Adaptor for Oscillating Intra Oral Blade does not come off by operating the Attachment.
- DO NOT operate the Attachment when the Cutting Accessory or Adaptor for Oscillating Intra Oral Blade is not mounted.
- Before mounting or removing the Cutting Accessory or Adaptor for Oscillating Intra Oral Blade, be sure to stop the Attachment completely.
- Mount the Cutting Accessory to the Attachment after mounting the Irrigation Nozzle.
- Be sure to use the NSK-specified Irrigation Nozzle (see "3-3 Part Names and Specifications").
For the related connection on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.


Operation of BMH Series

5-2-1 Bone Saw Attachment 200 (P200-REC)

Mounting the Irrigation Nozzle (Optional)

_ Mounting

- 1 Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- 2 Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).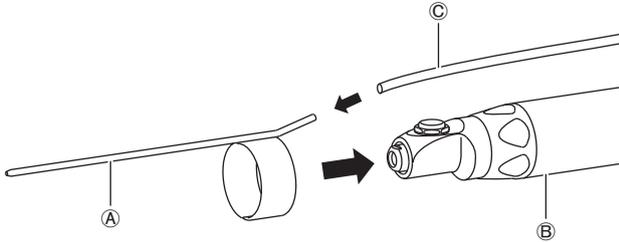

* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


Mounting and Removing the Cutting Accessories (Optional)

_ Mounting

- 1 Twist the shank of the Cutting Accessory (B) lightly with your fingers to insert it all the way while depressing the Fitting/Removal Button (A).
- 2 Release the Fitting/Removal Button (A) to set the Cutting Accessory in place.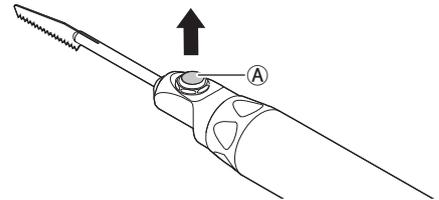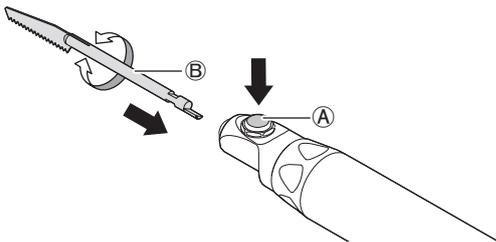

- 3 After mounting the Cutting Accessory (C), pull it lightly to confirm that it is securely connected to the Attachment.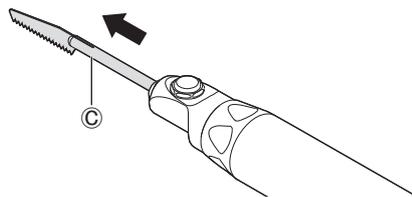

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.

_ Removing


Perform steps 1 and 2 of the mounting procedure in the reverse order.

5-2-2 Bone Saw Attachment 200 (P200-SAG)

Mounting the Irrigation Nozzle (Optional)

_Mounting

- ① Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- ② Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).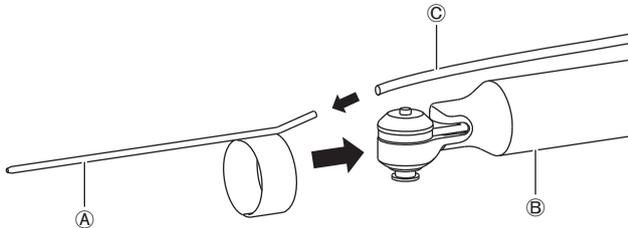

* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.


_Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


Mounting and Removing the Cutting Accessories (Optional)

_Mounting


- ① While depressing the Fitting/Removal Button (A), fit the Alignment Holes of a Cutting Accessory (B) into the convex part of the Attachment (C).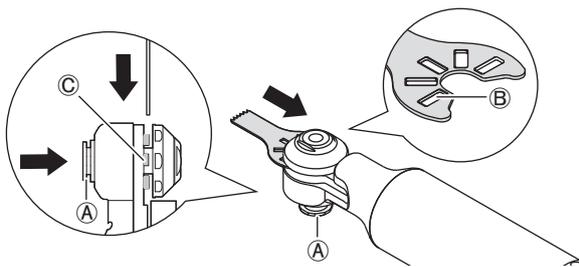


- ② Release the Fitting/Removal Button (A) to set the Cutting Accessory in place.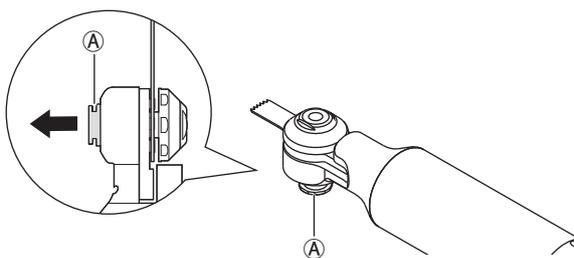

- ③ By twisting the Adjusting Nut (D) clockwise and counterclockwise, set the Cutting Accessory in place properly.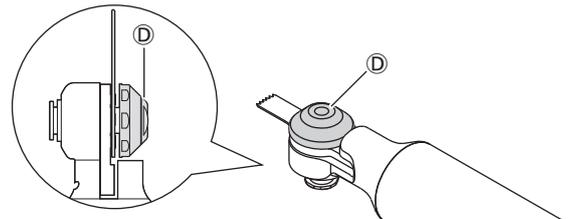

- ④ After mounting the Cutting Accessory, pull it lightly to confirm that it is securely connected to the Attachment.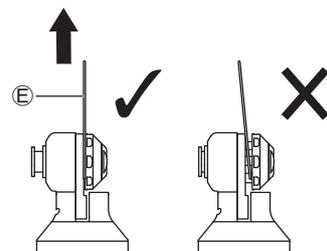

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.

_Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


Operation of BMH Series

5-2-3 Bone Saw Attachment 200 (P200-OSC)

Mounting the Irrigation Nozzle (Optional)

_Mounting

- 1 Insert the Irrigation Nozzle (A) onto the Bur Guard (B) of the Attachment, making sure that the correct end of the Irrigation Nozzle goes in.
- 2 Insert the Irrigation Tube (C) into the Irrigation Nozzle (A).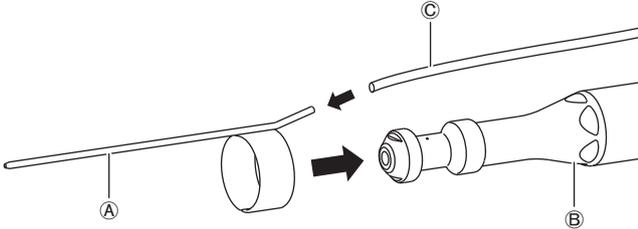


* For the related connection and the setting on the Control Unit, refer to OM-SE0021E Primado2 OPERATION MANUAL Control Unit and Foot Controls.


Mounting and Removing the Cutting Accessories (Optional)

_Mounting


- 1 While pulling down the Removal Lever (A), fit the Alignment Holes of a Cutting Accessory (B) into the convex part of the Attachment (C).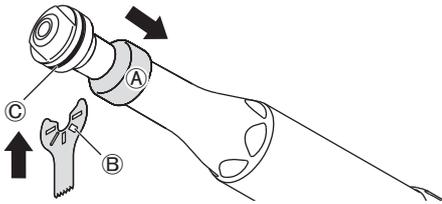


- 2 Release the Removal Lever (A) to set the Cutting Accessory in place.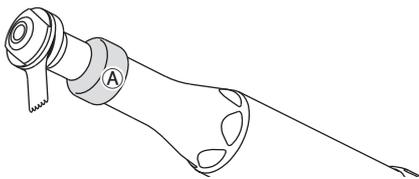

- 3 By twisting the Removal Lever (A) clockwise and counterclockwise, set the Cutting Accessory in place properly.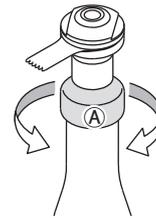

- 4 After mounting the Cutting Accessory (D), pull it lightly to confirm that it is securely connected to the Attachment.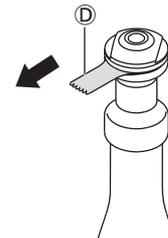

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.



_Removing

Perform steps 1 through 3 of the mounting procedure in the reverse order.


Mounting and Removing the Adaptor for Oscillating Intra Oral Blade (Optional)

_ Mounting

- 1 While pulling down the Removal Lever (A), fit the concave part of the Adaptor for Oscillating Intra Oral Blade (D) into the convex part of the Attachment (C).
- 3 By twisting the Removal Lever (A) clockwise and counterclockwise, set the Cutting Accessory in place properly.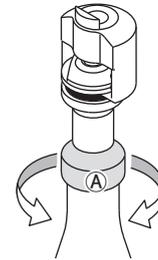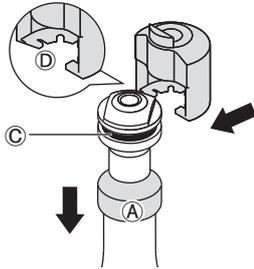

- 2 Release the Removal Lever (A) to set the Adaptor for Oscillating Intra Oral Blade in place.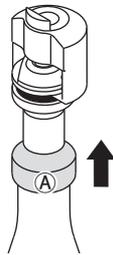



* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.

_ Removing


Perform steps 1 and 2 of the mounting procedure in the reverse order.

Mounting and Removing the Cutting Accessories (Optional)

- 1 Ensure the Screw part of the Adaptor for Oscillating Intra Oral Blade (F) and Notch Section on the Cutting Accessory (E) are located (12 o'clock) upward. Then insert the Cutting Accessory into the Adaptor for Oscillating Intra Oral Blade.
- 3 After mounting the Cutting Accessory (H), pull it lightly to confirm that it is securely connected to the Attachment.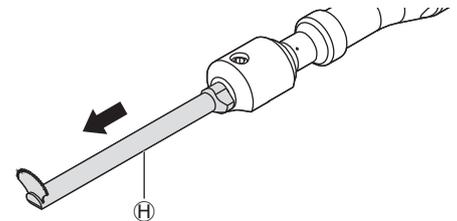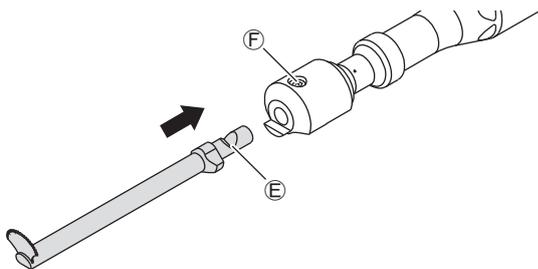

- 2 Rotate the Special Wrench (G) provided with the Adaptor in the side hole of the Adaptor to set the Cutting Accessory (H) in place.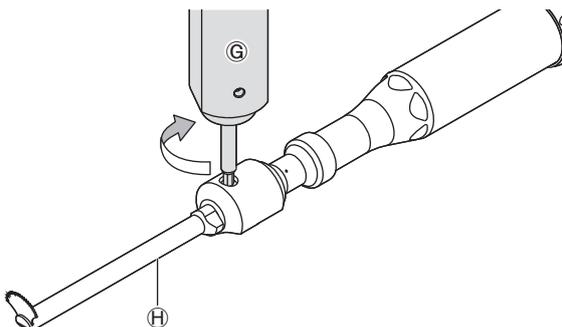

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.

_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.

6 Operation of WPD Series

6-1 Mounting and Removing the Attachment


⚠ CAUTION

- Be sure to slide the Safety Lock on the Hand Switch of the Motor to the OFF position when mounting or removing the Attachment.
- Refer to the Operation Manual of the Motor.
- Remove the Cutting Accessory before mounting or removing the Attachment to prevent personal injury.
- After mounting the Attachment, make sure that it is correctly connected to the Motor.

6-1-1 WPD Series Attachment

_ Mounting

Align the arrow marks of the Attachment (A) and the Motor (B), then push the Attachment in until it clicks.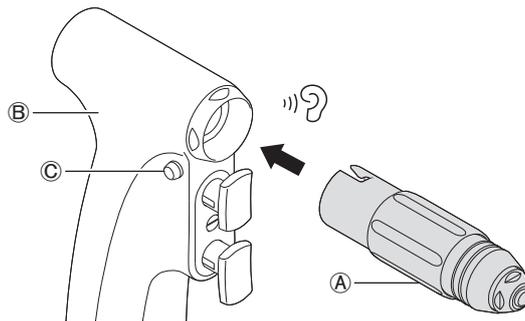

_ Removing

Pull the Attachment (A) straight out while pressing the Release Button of the Motor (C).

6-2 Mounting and Removing the Cutting Accessories (Optional)

⚠ WARNING

- Be sure to use the NSK recommended Cutting Accessory for each Attachment (see "3-3 Part Names and Specifications"). When using an unrecommended product, it cannot be firmly fixed. This may lead to the product coming off during the surgical operation.
- DO NOT rotate the Attachment if the chuck of the Attachment is open. Doing so may cause thermal injury and/or damage to the product due to heat generation of the Attachment.


⚠ CAUTION

- After mounting the Cutting Accessory and locking the chuck, pull it lightly to confirm that it is securely connected to the Attachment.
- Before use, confirm that the Cutting Accessory does not come off by operating the Attachment.
- DO NOT operate the Attachment when the Cutting Accessory is not mounted.
- Before mounting or removing the Cutting Accessory, be sure to stop the Attachment completely.


6-2-1 Primado Wire Pin Driver Attachment (PD-DA-W/P)

_ Mounting


- ① Insert the wire (or pin) ② through the tip of the Attachment straight without twisting, while the Attachment Lever ① is released.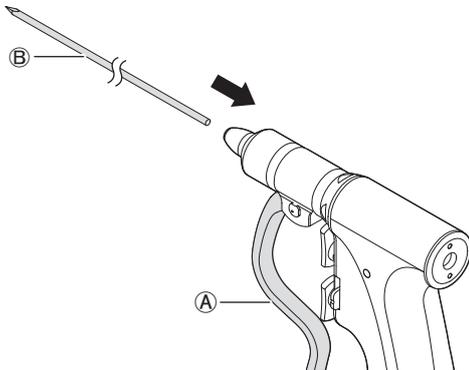

- ③ After mounting the wire (or pin) ②, pull it lightly to confirm that it is securely connected to the Attachment.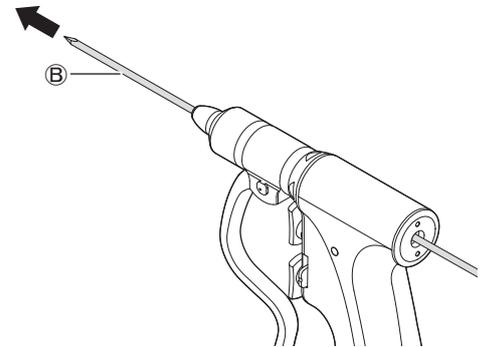

- ② Adjust the length of the wire (or pin) ② as required and pull the Attachment Lever ①. The wire (or pin) will be set in place.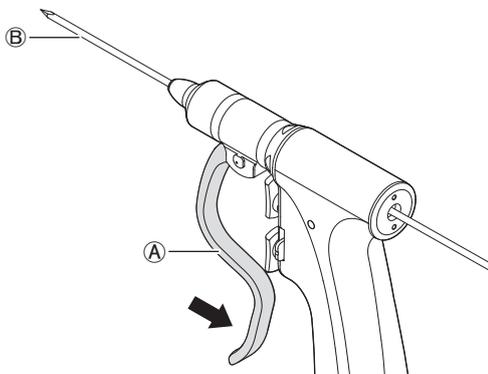

_ Removing



Perform steps 1 and 2 of the mounting procedure in the reverse order.

Operation of WPD Series


6-2-2 Primado Wire Pin Driver Attachment (PD-DA-J4/J6)

_ Mounting

- 1 Rotate the T wrench (A) in the direction of (B), insert the drill bit, etc. (C) through the tip of the Attachment and adjust its length.
- 3 After mounting the drill bit, etc. (C), pull it lightly to confirm that it is securely connected to the Attachment.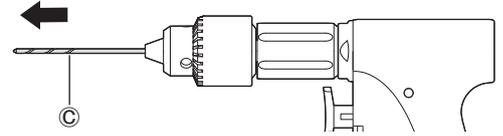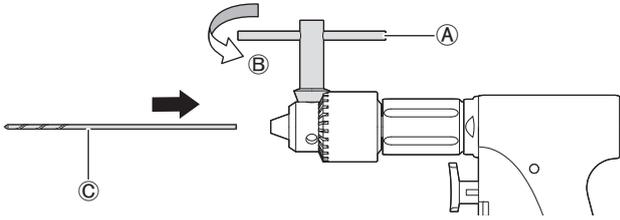

- 2 Tighten the screw on the tip of the Attachment (3 places) equally by rotating the T wrench (A) in the direction of (D).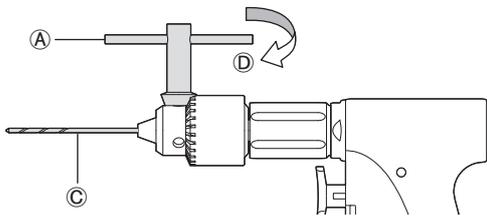



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.

6-2-3 Primado Wire Pin Driver Attachment (PD-DA-A)



_ Mounting

- 1 Insert the commercially available AO-type drill bit (A) while twisting it with your fingers through the tip of the Attachment.
- 3 Release the Removal Cover (B) to set the commercially available AO-type drill bit (A) in place.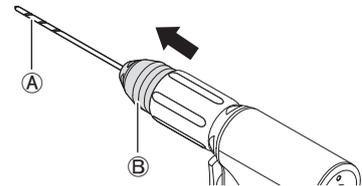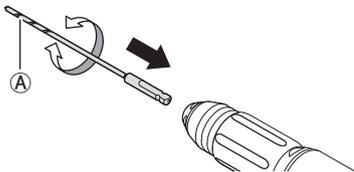

- 2 Slide the Removal Cover (B) and insert the commercially available AO-type drill bit (A) further while twisting it with your fingers.

- 4 After mounting the commercially available AO-type drill bit (A), pull it lightly to confirm that it is securely connected to the Attachment.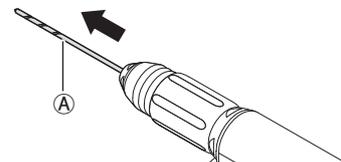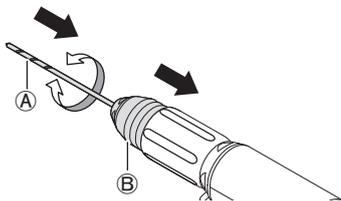

_ Removing

Perform steps 1 through 3 of the mounting procedure in the reverse order.


6-2-4 Primado Wire Pin Driver Attachment (PD-DA-K)

_ Mounting

- 1 Rotate the Twist Collet (A) in the direction of (B), insert the drill bit, etc. (C) through the tip of the Attachment and adjust its length.
- 2 Rotate the Twist Collet (A) in the direction of (D) to set the drill bit, etc. (C) in place.
- 3 After mounting the drill bit, etc. (C), pull it lightly to confirm that it is securely connected to the Attachment.

- 2 Rotate the Twist Collet (A) in the direction of (D) to set the drill bit, etc. (C) in place.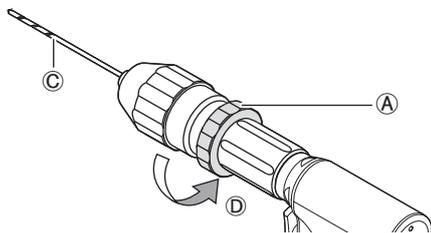



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


6-2-5 Primado Wire Pin Driver Attachment (PD-RA-A/M/H/T)

_ Mounting

- 1 Align the shank of the reamer bit with Hudson type chuck (B) with the groove of the Attachment (C) while pulling the Mounting/Removal Cover (A), and push it all the way.
- 2 Release the Mounting/Removal Cover (A) to set the reamer bit in place.
- 3 After mounting the reamer bit (D), pull it lightly to confirm that it is securely connected to the Attachment.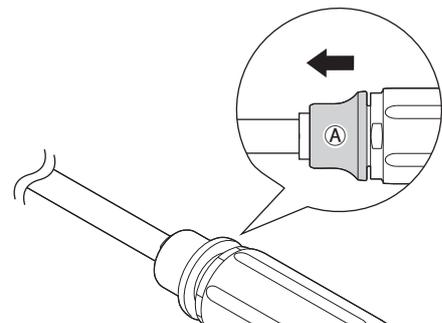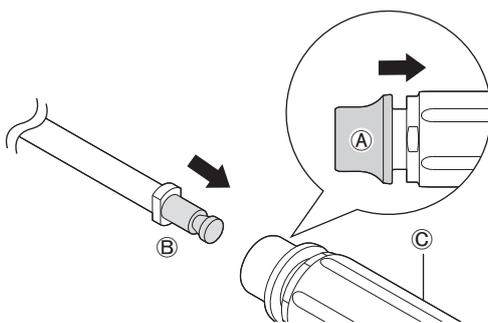

- 3 After mounting the reamer bit (D), pull it lightly to confirm that it is securely connected to the Attachment.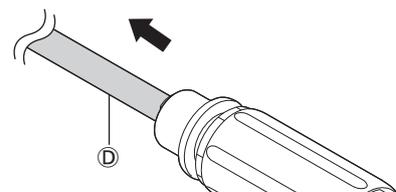



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


6-2-6 Primado Wire Pin Driver Attachment (PD-SA-R)

_ Mounting

- ① Twist the shank of the Cutting Accessory (B) lightly with your fingers to insert it all the way while depressing the Mounting/Removal Button (A).
- ② Release the Mounting/Removal Button (A) to set the Cutting Accessory in place.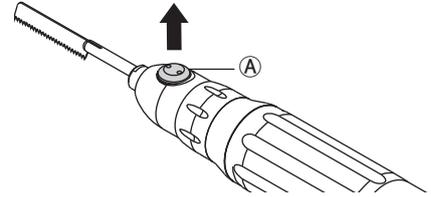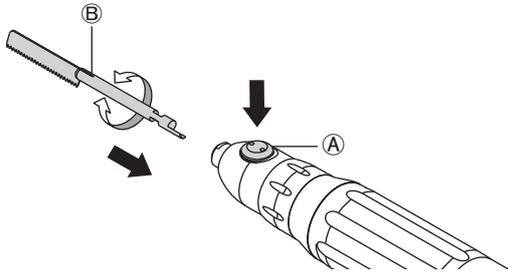

- ③ After mounting the Cutting Accessory (C), pull it lightly to confirm that it is securely connected to the Attachment.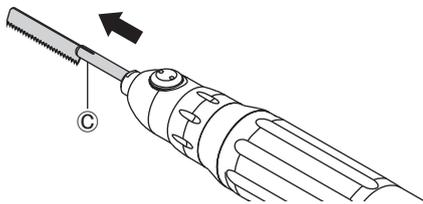

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.



_ Removing

Perform steps 1 and 2 of the mounting procedure in the reverse order.


6-2-7 Primado Wire Pin Driver Attachment (PD-SA-S)

_ Mounting


- ① While depressing the Mounting/Removal Button (A), fit the Alignment Holes of a Cutting Accessory (B) into the convex part of the Attachment (C).
- ③ By twisting the Adjusting Nut (D) clockwise and counterclockwise, set the Cutting Accessory in place properly.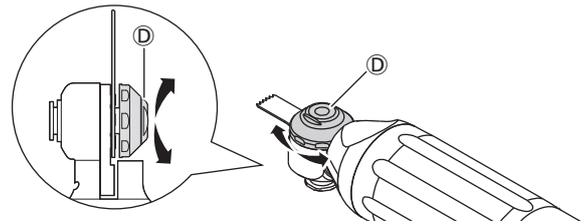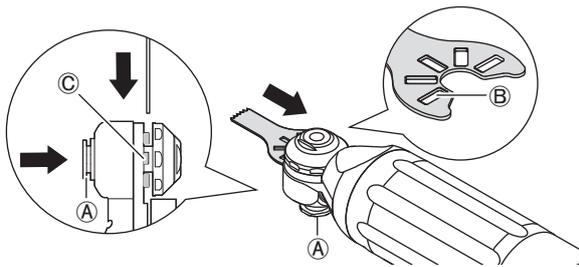

- ② Release the Mounting/Removal Button (A) to set the Cutting Accessory in place.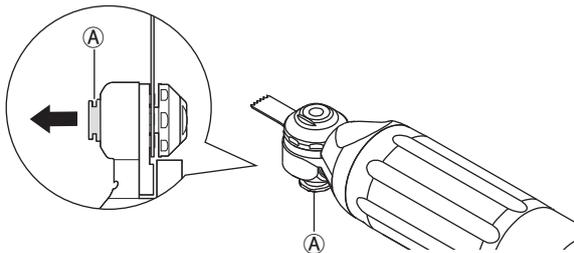

- ④ After mounting the Cutting Accessory, pull it lightly to confirm that it is securely connected to the Attachment.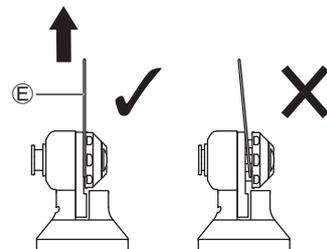

* If the Cutting Accessory is not set in place properly, repeat the procedure from 1 - 3 to set it in place properly.

_ Removing


Perform steps 1 and 2 of the mounting procedure in the reverse order.

7 Mounting and Removing the Bur Guard

Applicable Attachment	Slim Handpiece 200 Super Slim Handpiece 200
-----------------------	--



Removing

- 1 Place the Wrench for Slim/Super Slim (P200-1ARA-W) (A) onto the Removal Ring (B), loosen the Removal Ring (B) by rotating it in the direction of (C) and pull the Bur Guard (D) straight out.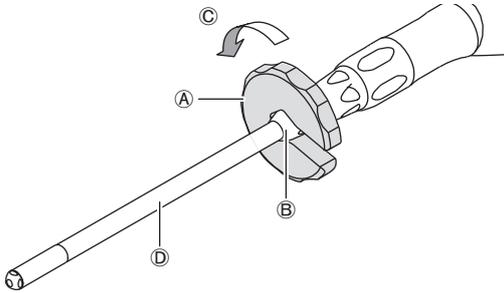

Mounting

- 1 Align the Bur Guard (D) and Attachment (E) and tighten the Removal Ring (B) manually.
- 2 Rotate the Removal Ring (B) in the direction of (F) with the Wrench for Slim/Super Slim (P200-1ARA-W) (A) to fix it firmly.

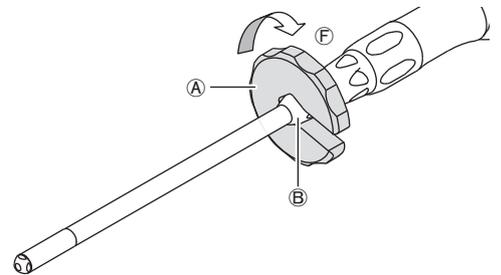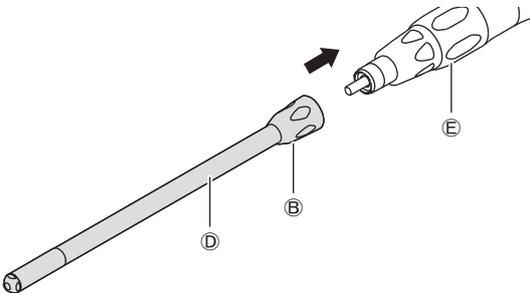8 Check before Operation

Conduct check before operation according to the following procedure to confirm that there are no abnormalities. If any abnormality is observed, perform "11 Troubleshooting".

- 1 Attach the Attachment to the Motor, then attach the Cutting Accessory to the attachment. Refer to Check before Operation of each Attachment.
- 2 Check for abnormalities at the connection part of the Attachment and Cutting Accessory.
- 3 Operate the Attachment at the rotational speed permissible for the Cutting Accessory mounted. Confirm that nothing is loose and that no abnormal vibration, heating or noise occurs. If any abnormality is observed, stop use and perform "11 Troubleshooting".
- 4 Ensure that there are no abnormalities before use.

9 Periodical Maintenance Checks

Every three months, perform periodical maintenance checks on this product according to the following list. If any abnormalities are found, contact your Authorized NSK Dealer.

Checking Item	Description
Outer Surface	Check for any scratches, damage or abnormalities on the outer surface.
Each Connection Part	Check for any scratches, damage or deformation of each connection part.
Attachment	Connect the Attachment and check for abnormalities at the connection part.
Rotation	After connecting each part, fasten the Twist Collet, close the Latch (Chuck Close position) and rotate the Motor for approximately 1 minute by pressing the Foot Control Pedal or Hand Switch of the Motor. Check for any abnormal vibration, noise or heat generation. Check that the Motor stops normally after releasing the Foot Control Pedal or Hand Switch of the Motor.

10 Reprocessing

Perform reprocessing appropriately according to OM-SZ0911MA Primado2 REPROCESSING MANUAL.

11 Troubleshooting

If you suspect any faults, check the following troubleshooting guide prior to asking for repair work. If none of the cases below apply or if the situation does not improve despite your efforts or measures, the damage could be serious; call your Authorized NSK Dealer to fix the problem.

11-1 SMH Series Attachment

Symptoms	Causes/Points of checking	Countermeasures
The Attachment cannot be connected to the Motor.	The '▶' mark of the Attachment and '●' mark of the Motor are not aligned.	Connect the Attachment correctly (see "4-1 Connecting and Disconnecting the Attachment").
	Connection part has been deformed.	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be firmly mounted.	The Cutting Accessory has not been inserted deep enough.	Insert the Cutting Accessory all the way through.
	The Twist Collet or Latch is in 'OPEN' (Chuck Open position).	After inserting the Cutting Accessory, fasten the Twist Collet or slide the Latch (Chuck Close position).
	The Chuck is closed.	Check the procedure of Mounting and Removing Cutting Accessories, etc.
	Incorrect Cutting Accessory is mounted.	Check the applicable Cutting Accessory.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	Twist Collet has been damaged or deformed.	
The exposure length of the Cutting Accessory cannot be adjusted. (Slim Attachment 300, Standard Attachment 300)	Twist Collet has been damaged or deformed.	Contact your Authorized NSK Dealer.
No rotation when the Hand Switch is depressed.	Safety Lock of the Motor is "OFF".	Slide the Safety Lock of the Motor to the ON position. – In case of no rotation when the Safety Lock is in ON position, there may be a malfunction. Contact your Authorized NSK Dealer.
No rotation when the Cutting Accessory is securely mounted.	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.

Troubleshooting

Symptoms	Causes/Points of checking	Countermeasures
Heat generation during operation.	The Twist Collet or Latch is in ' OPEN ' (Chuck Open position).	After inserting the Cutting Accessory, fasten the Twist Collet or slide the Latch (Chuck Close position).
	The ' ▶ ' mark of the Attachment and ' ● ' mark of the Motor are not aligned.	Align the ' ▶ ' mark of the Attachment and the ' ● ' mark of the Motor, then push the Attachment in until it clicks to connect firmly.
	Continuous use over a long period.	Stop using the Attachment and the Motor immediately. Wait until the Attachment and Motor cool down before using the Attachment and Motor again.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	The blade part of the Cutting Accessory mounted has worn out.	
<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.	
Abnormal vibration and noise. Cutting Accessory center run-out.	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be removed when operating the Twist Collet, Latch, Removal Lever or Release Button.	The Chuck has been damaged.	
	Incorrect Cutting Accessory is mounted.	
The Attachment cannot be removed from the Motor.	The Cutting Accessory mounted has been deformed.	Remove the Attachment correctly (see "4-1 Connecting and Disconnecting the Attachment").
	Connection part has been deformed.	
	The Attachment is not removed the correctly.	

11-2 HMH Series Attachment

Symptoms	Causes/Points of checking	Countermeasures
The Attachment cannot be connected to the Motor.	The Alignment Pin is not in position.	Connect the Attachment correctly (see "4-1 Connecting and Disconnecting the Attachment").
	Connection part has been deformed.	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be firmly mounted.	The Cutting Accessory has not been inserted deep enough.	Insert the Cutting Accessory all the way through.
	The Twist Collet or Latch is in ' OPEN ' (Chuck Open position).	After inserting the Cutting Accessory, fasten the Twist Collet or slide the Latch (Chuck Close position).
	The Chuck is closed.	Check the procedure of Mounting and Removing Cutting Accessories, etc.
	Incorrect Cutting Accessory is mounted.	Check the applicable Cutting Accessory.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	Twist Collet has been damaged or deformed.	Contact your Authorized NSK Dealer.
No rotation when the Hand Switch is depressed.	Safety Lock of the Motor is " OFF " .	Slide the Safety Lock of the Motor to the ON position. – In case of no rotation when the Safety Lock is in ON position, there may be a malfunction. Contact your Authorized NSK Dealer.
No rotation when the Cutting Accessory is securely mounted.	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
Heat generation during operation.	The Twist Collet or Latch is in ' OPEN ' (Chuck Open position).	After inserting the Cutting Accessory, fasten the Twist Collet or slide the Latch (Chuck Close position).
	Continuous use over a long period.	Stop using the Attachment and the Motor immediately. Wait until the Attachment and Motor cool down before using the Attachment and Motor again.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	The blade part of the Cutting Accessory mounted has worn out.	
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.

Troubleshooting

Symptoms	Causes/Points of checking	Countermeasures
Abnormal vibration and noise. Cutting Accessory center run-out.	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be removed when operating the Twist Collet, Latch or Removal Lever.	The Chuck has been damaged.	
	Incorrect Cutting Accessory is mounted.	
	The Cutting Accessory mounted has been deformed.	
The Attachment cannot be removed from the Motor.	Connection part has been deformed.	Contact your Authorized NSK Dealer.
	The Attachment is not removed the correctly.	Remove the Attachment correctly (see "4-1 Connecting and Disconnecting the Attachment").

11-3 BMH Series Attachment

Symptoms	Causes/Points of checking	Countermeasures
The Attachment cannot be connected to the Motor.	The Alignment Pin and Alignment Hole are not in position.	Connect the Attachment correctly (see "5-1 Connecting and Disconnecting the Attachment").
	Connection part has been deformed.	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be firmly mounted.	The Cutting Accessory has not been inserted deep enough.	Insert the Cutting Accessory all the way through.
	The Chuck is closed.	Check the procedure of Mounting and Removing Cutting Accessories, etc.
	Incorrect Cutting Accessory is mounted.	Check the applicable Cutting Accessory.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
The blade part of the Cutting Accessory mounted has worn out.		
No rotation when the Hand Switch is depressed.	Safety Lock of the Motor is " OFF " .	Slide the Safety Lock of the Motor to the ON position. – In case of no rotation when the Safety Lock is in ON position, there may be a malfunction. Contact your Authorized NSK Dealer.
No rotation when the Cutting Accessory is securely mounted.	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.

Symptoms	Causes/Points of checking	Countermeasures
Heat generation during operation.	Continuous use over a long period.	Stop using the Attachment and the Motor immediately. Wait until the Attachment and Motor cool down before using the Attachment and Motor again.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	The blade part of the Cutting Accessory mounted has worn out.	
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
Abnormal vibration and noise. Cutting Accessory center run-out.	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be removed when operating the Removal Lever or Fitting/Removal Button.	The Chuck has been damaged.	
	Incorrect Cutting Accessory is mounted.	
	The Cutting Accessory mounted has been deformed.	
The Attachment cannot be removed from the Motor.	Connection part has been deformed.	Remove the Attachment correctly (see "5-1 Connecting and Disconnecting the Attachment").
	The Attachment is not removed the correctly.	

Troubleshooting

11-4 WPD Series Attachment

Symptoms	Causes/Points of checking	Countermeasures
The Attachment cannot be connected to the Motor.	The mounting position is not correct.	Connect the Attachment correctly (see "6-1 Mounting and Removing the Attachment").
	The Attachment has not been inserted deep enough.	Insert the Attachment all the way through.
	Connection part has been deformed.	Contact your Authorized NSK Dealer.
The Cutting Accessory cannot be firmly mounted.	The Chuck is closed.	Check the procedure of Mounting and Removing Cutting Accessories, etc.
	Incorrect Cutting Accessory is mounted.	Check the applicable Cutting Accessory.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
No rotation when the Hand Switch is depressed.	Safety Lock of the Wire Pin Driver is " OFF ".	Slide the Safety Lock of the Wire Pin Driver to the ON position. – In case of no rotation when the Safety Lock is in ON position, there may be a malfunction. Contact your Authorized NSK Dealer.
No rotation when the Cutting Accessory is securely mounted.	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
Heat generation during operation.	Continuous use over a long period.	Stop using the Attachment and the Motor immediately. Wait until the Attachment and Motor cool down before using the Attachment and Motor again.
	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	The blade part of the Cutting Accessory mounted has worn out.	
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.
Abnormal vibration and noise. Cutting Accessory center run-out.	The Cutting Accessory mounted has been deformed.	Replace the Cutting Accessory with a new one.
	<ul style="list-style-type: none"> ● The bearing has overheated. ● Foreign materials inside. ● The bearing has worn out. 	Contact your Authorized NSK Dealer.

Symptoms	Causes/Points of checking	Countermeasures
The Cutting Accessory cannot be removed when operating the Mounting/Removal Cover or Mounting/Removal Button.	The Chuck has been damaged.	Contact your Authorized NSK Dealer.
	Incorrect Cutting Accessory is mounted.	
	The Cutting Accessory mounted has been deformed.	
The Attachment cannot be removed from the Wire Pin Driver.	Connection part has been deformed.	Remove the Attachment correctly (see "6-1 Mounting and Removing the Attachment").
	The Attachment is not removed the correctly.	

12 Warranty

Our products are subjected to rigorous quality control and inspections. In the unlikely event of a product proving to be faulty within the warranty period when it has been used normally, it will be repaired free of charge subject to the terms and conditions below. For such repair requests, please contact your Authorized NSK Dealer from whom the product was purchased.

Warranty period.....One year

- In the following cases, the repair has to be paid for even if the product is within the warranty period:
 - Fault and damage caused by misuse, or inappropriate repair or modification.
 - Fault and damage caused during transit or dropping after the product was purchased.
 - Fault and damage attributable to fire, earthquake, flooding, power surge, environmental pollution or other acts of providence.
 - Fault and damage caused by the use of consumables or methods other than those specified by NSK.
 - When the product has been repaired using components other than genuine components supplied by NSK.
- For repairs after the warranty period, please contact your Authorized NSK Dealer from whom the product was purchased.

13 Optional Accessories

Product Name	Order Code	Quantity	Remarks
Cleaning Tube 6.0/L650	PD-CTTU-606501	1	-
Cleaning Tube 6.0/L50 (4 pcs.)	PD-CTTU-600504	1	-
Cleaning Tube 1.5/L25 (2 pcs.)	PD-CTTU-150252	1	-
Cleaning Holder	PD-CTH	1	Cleaning Tube 6.0/L650 and Cleaning Tube 6.0/L50 (4 pcs.) are included with this part.
Cleaning Adaptor Luer M	PD-CTAD-LM	1	-
Cleaning Adaptor SM	PD-CTAD-SM	1	-
Cleaning Adaptor 1T	PD-CTAD-1T	1	-
Cleaning Adaptor HM	PD-CTAD-HM	1	-
Cleaning Adaptor WPD	PD-CTAD-WPD	1	-
Cleaning Adaptor IN	PD-CTAD-IN	1	Cleaning Tube 1.5/L25 (2 pcs.) is included with this part.
Cleaning Adaptor G	PD-CTAD-G	1	-
PANA SPRAY Plus	Z182100	1	-
PANA SPRAY Plus (6 pcs.)	Y900630	1	-
EZ Spray Adaptor	PD-EZ-M	1	For PANA SPRAY Plus
Spray Adaptor for Slim	P200-EZ-SM	1	Slim Handpiece 200 Super Slim Handpiece 200 Standard Handpiece 200 Drill Handpiece 200 Metal Cutting Handpiece 200 ContraAngle Attachment 200 Perforator Handpiece 200 Craniotome Handpiece 200 Slim Attachment 300 Standard Attachment 300
Spray Adaptor for Bone Saw	P200-EZ-BM	1	Bone Saw Attachment 200
Spray Adaptor	P200-EZ-WPD	1	Primado Wire Collet
Irrigation Nozzle	P200-IN-1A155	1	Slim Handpiece 200
	P200-IN-1A175	1	Slim Attachment 300
	PD-IN-D	1	Standard Handpiece 200 Standard Attachment 300
	PD-IN-2ES	1	
	PD-IN-2S	1	Drill Handpiece 200
	PD-IN-2M	1	Primado Drill Attachment
	PD-IN-3S	1	High Speed Handpiece 200
	PD-IN-3M	1	
	PD-IN-3L	1	
	P200-IN-BS	1	Bone Saw Attachment 200
STD Attachment Beak	P300-2-B	1	Standard Attachment 300
Slim Tube Hood	P300-1T-H	1	Slim Attachment 300
Slim Tube Beak	P300-1T-B	1	
Wrench for Slim/Super Slim	P200-1ARA-W	1	Slim Handpiece 200 Super Slim Handpiece 200
Adaptor for Oscillating Intra Oral Blade	PD-OSC-A	1	Bone Saw Attachment 200 (only for Primado2 Osc. Attachment)
Special Wrench (PD-OSC Wrench)	SZ114011	1	Adaptor for Oscillating Intra Oral Blade
T wrench	PD-TW-J4	1	Primado Wire Pin Driver Attachment (for Jacobs Keyed Drill Chuck)
	PD-TW-J6	1	

14 Disposing of Product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose of the product for you.

15 Symbols

	Manufacturer		Catalog number (Order Code)
	Conforms to CE European Directive of "Medical device directive 93/42/EEC".		State of being fixed
	Authorized representative in the European community.		State of being unfixed
	Date of manufacture		Listen (Clicking sound)
	See operation manual.		Hand Mode
	Caution, Consult operation instructions.		Foot Mode
	GS1 DataMatrix for Unique Device Identifier.		Temperature limitation
	This product can be cleaned using a washer-disinfector.		Humidity limitation
	This product can be sterilized in a steam sterilizer at 135°C.		Atmospheric pressure limitation
	Serial No.		

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NSK Europe GmbH